

Rolyan® Elbow Pad:
A731-5 and A731-500

INDICATIONS

- To help protect elbows when they rest on a hard surface

CONTRAINDICATIONS

- Not in the presence of elbow fractures and dislocations
- Not in the presence of open wounds, infected areas, and blisters

PRECAUTIONS

- Change position and check the elbow every 20 minutes during use.
- If unusual swelling, skin discoloration or discomfort occurs, discontinue use.
- Do not strap the pad in place for protection.

CARE AND CLEANING

Remove the outer fabric cover and clean frequently. Wash cover with mild soap and warm water and allow to air dry thoroughly before replacing cover on the pad. Surface clean the gel pad using a damp cloth or cold disinfectant and allow it to air dry completely before reuse.

INSTRUCTIONS FOR USE

Place the pad on the counter or table top where the elbow rests.



Patterson Medical
1000 Remington Blvd.
Suite 210
Bolingbrook, IL 60440
Phone: 800-323-5547
Direct Phone: 630-378-6000
Fax: 800-547-4333
Direct Fax: 630-378-6310

homecraft
AbilityOne

Authorized European Representative
Homecraft
An AbilityOne Company
Shelley Road, Lowmoor Business Park
Kirkby-in-Ashfield, Nottinghamshire NG17 7ET
United Kingdom
Phone: 1623-726928
Fax: + 44 (0) 1623 755585