
## Rolyan® Wedge Flex® Abduction Pillow:

A513-500 and A513-501

A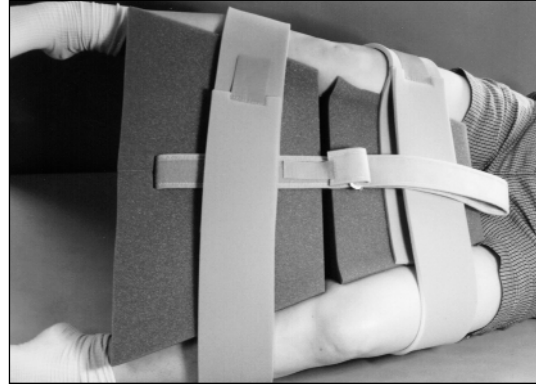

B

### INDICATIONS

- For positioning following hip surgery
- Can also be used for people whose skin between the legs has a tendency to break down

### CONTRAINDICATIONS

- Not for people unable to understand instructions unless closely monitored by a healthcare professional

### PRECAUTIONS

- **This product is to be fitted initially by a healthcare professional who is familiar with the purpose for which it is prescribed. The healthcare professional is responsible for providing wearing instructions and precautions to other healthcare practitioners, care providers involved in the patient's care, and the patient.**
- **If unusual swelling, skin discoloration or discomfort occurs, use should be discontinued and a healthcare professional consulted.**

- **Be careful that the straps are not so tight that they interfere with circulation.**

### CARE AND CLEANING

- The wedge can be covered with a towel or pillowcase during use.
- If cleaning is necessary, remove the central strap from the wedge. The strap can be machine washed in warm water and machine dried at low temperature. Be sure it is completely dry before reapplying the wedge.
- The foam wedge is open-cell foam, and it should not be allowed to get wet. It can be wiped with a damp cloth.

### INSTRUCTIONS FOR USE

1. The foam piece can be modified using an electric knife if necessary. The slit can be deepened to allow greater knee flexion (deepening the slit may also promote increased hip flexion). In addition, the position of the slit relative to the proximal or distal edge can be adjusted by trimming the

proximal or distal edge. Avoid cutting close to the straps.

2. To apply the wedge, open the proximal and distal leg straps.
3. With the patient in a supine position, put the wedge between the legs with the loop on the central strap facing up and the knees even with the slit in the wedge (A).
4. Fasten the proximal and distal straps around the legs (A).
5. For patient repositioning, the loop strap may be pulled gently to help move the legs.
6. To bend the knees slightly while sidelying, loosen the central loop strap, which controls the amount of flexion allowed, and bend the wedge along the slit to achieve an appropriate knee angle (B). Tighten the loop strap. Use pillows or a body wedge for support behind the patient.

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**Patterson Medical Ltd.**