#### INDICATIONS

- To help correct ulnar deviation
- With wrist stay, to help support the wrist

#### CONTRAINDICATIONS

- Not for use over open wounds and infected areas

#### PRECAUTIONS

- **This splint is to be fitted initially by a healthcare professional who is familiar with the purpose for which it is prescribed. The healthcare professional is responsible for providing wearing instructions and precautions to other healthcare practitioners, care providers involved in the patient's care, and the patient.**
- **If unusual swelling, skin discoloration or discomfort occurs, use should be discontinued and a healthcare professional consulted.**
- **Be careful that the splint is not so tight that it interferes with circulation.**
- **After heating splinting material, always check its temperature before applying it to the patient.**

#### CARE AND CLEANING

- The stay will lose its shape in temperatures over 135°F (57°C). It should be kept away from sources of heat such as ovens, hot water, open flames, and sunny car windows.
- Remove stay before washing splint.
- Fasten hook-and-loop fasteners. Hand wash fabric in warm water and mild soap. Rinse thoroughly. Do not

wring. Dry at room temperature. Be sure the splint is completely dry before reapplying it.

#### INSTRUCTIONS FOR USE


Note that the hook material on the splint faces away from the skin. The soft splint and optional wrist stay are shown in **A**.

#### Molding the optional wrist stay

1. Heat the stay, which is made from **Polyform**® splinting material, in water heated to 150° to 160°F (65°–70°C) for one minute or until pliable.
2. Remove from heat pan and place on a towel on a flat surface.
3. Mold the stay on the volar wrist surface with the cutout area around the thenar eminence. Roll back the material to clear the thenar eminence and the distal palmar crease.
4. Put the cooled stay into the pocket.

#### Applying the splint

5. Apply the splint and fasten the D-ring wrist straps.
6. Wrap the first finger separator between the index and middle fingers and around the index finger, attaching it to the hook material on the splint with a gentle ulnar-to-radial pull.
7. Continue wrapping the finger separators around the fingers, attaching them to the hook on the previous finger separator, until all of them are fastened.
8. A completed splint is shown in **A**.

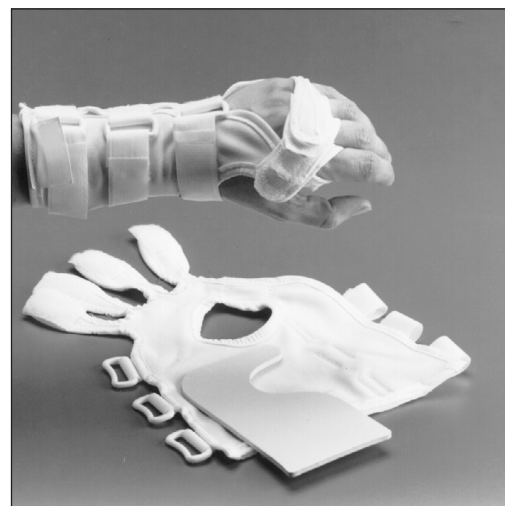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**Patterson Medical Ltd.**