

Patterson Medical® Cold Packs:

3545, 3547, 3548-A, 3548-B, 3548-C, 3548-D



APPLICATION

- For initial chilling of the pack, place flat in chilling unit freezer for 2 hours. Do not store anything on top of the pack. Do not chill below 0°F (-17.8°C).
- Remove pack from freezer and place a hand towel between the skin and the pack to avoid frostbite. Apply to area being treated.
- Treatment time should not exceed 30 minutes.

PRECAUTIONS

- Do not sit, lie or sleep on the pack.
- Constantly monitor the pack application to insure that the patient's skin is not being irritated by the cold.
- Do not attempt to use the pack as a hot pack. No hot water. No microwave. Use only as a cold therapy.

CARE INSTRUCTIONS

- Wipe all moisture from the pack before returning to freezer.
- It is best not to fold the pack for storage in freezer.
- Packs are most serviceable when kept in a freezer set at 12°F (-11.1°C).
- Store in freezer when not in use.
- Caution: If the cold pack is dropped, it should be checked for any damage/leaks prior to use.

ESPAÑOL

Compresas Frías Patterson

INSTRUCCIONES PARA EL USO DE LA COMPRESA:

- Para enfriar la compresa, colóquela en el congelador por 2 horas. No ponga nada encima de la compresa. No la congele bajo 0°F (-17.8°C).
- Saque la compresa del congelador y ponga una toalla entre la piel y la compresa para evitar la congelación. Aplicar la compresa en la zona afectada.
- Se recomienda una duración del tratamiento que no sobrepase los 30 minutos.

PRECAUCIONES:

- No se siente, se acueste, o duerma en la compresa
- Controle la aplicación de la compresa para asegurar que la piel no se irrite por el frío
- No intente usar la compresa como compresa caliente. No la use con agua caliente ni con microondas. Úsela exclusivamente para aplicación de terapia fría.

INSTRUCCIONES PARA EL CUIDADO DE LA COMPRESA:

- Seque la compresa de toda humedad antes de regresarla al congelador
- Cuando se almacene en el congelador, no doble la compresa.
- Las compresas sirven mejor cuando se mantienen en el congelador a una temperatura de 12°F (-11.1°C).
- Guarde en el congelador cuando no se utilicen.
- Precaución: Si la compresa fría se cae, revisela para daños/fugas antes de usar.



Distributed by:
Patterson Medical
28100 Torch Parkway, Ste. 700
Warrenville, IL 60555, USA
Phone: 800-323-5547
Direct Phone: 630-393-6000
Fax: 800-547-4333
Direct Fax: 630-393-7600

Manufactured for:
Patterson Medical Supply Inc.
W68 N158 Evergreen Blvd.
Cedarburg, WI 53012, USA
Phone: 800-323-5547
Fax: 800-547-4333

Patterson Medical Canada
6675 Millcreek Drive, Unit 3
Mississauga, ON L5N 5M4
Phone: 800-665-9200
Direct Phone: 905-858-6000
Direct Fax: 905-858-6001
Toll-Free Fax: 877-566-8611

Authorized European Representative
Patterson Medical, Ltd.
Nunn Brook Road
Huthwaite, Sutton-in-Ashfield
Nottinghamshire, NG17 2HU, UK
International
Phone: +44 (0) 1623 448 706
Fax: +44 (0) 1623 448 784