


Personal Alarm

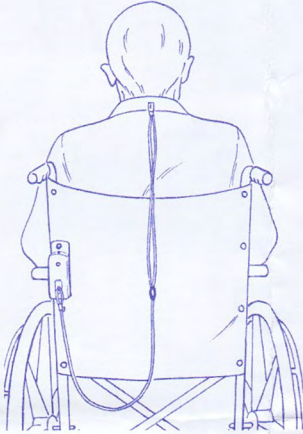

Reorder	Description	Unit
909211	Personal Alarm for Wheelchair and Bed	Each

Purpose

Skil-Care's Personal Alarm alerts caregivers when a resident/patient has left or fallen from the bed or wheelchair.

Cautions



1. The Personal Alarm is not a substitute for proper supervision. Fall-risk and wander-risk residents/patients should be monitored regularly.
2. Check battery each time alarm is used. This is done by setting alarm in high or low volume position and removing magnet. If alarm is weak or fails to sound, replace the 9-volt battery.
3. LED should glow when the alarm is turned on (high or low). If LED fails to glow, replace battery.
4. Periodically check clip to make certain it is attached to resident's/patient's clothing. Patients/residents may be able to defeat the purpose of the alarm by removing the clip.
5. Periodically check to make certain that patient/resident has not removed battery.

Wheelchair Installation

You may choose one of the following two ways to secure the alarm to the wheelchair.


Mounting Bracket

1. Remove the second-from-top screw from either side of the wheelchair backrest. (Illustration C)
2. Remove grommet from screw, place screw through hole in alarm mounting bracket, and refasten screw to wheelchair backrest. (Illustration D)

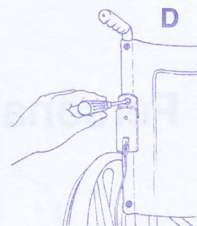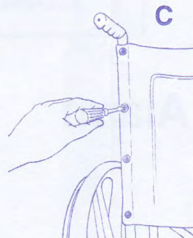

Cord Loop

1. Insert enclosed cord loop through bracket hole. (Illustration E)
2. Hang from wheelchair push-handle. (Illustration F)

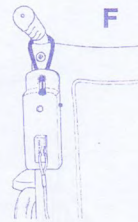

Wheelchair Use

Adjust cord to desired length and attach to patient's/resident's whirt or blouse as shown in illustration A. Set volume at high or low.

Bed Installation and Use

1. The Personal Alarm is attached to headboard by means of Velcro® strips (included). Remove backing from one strip and apply it to back of alarm.
2. Peel backing from other Velcro® strip and attach it to the upper left or right side of headboard.
3. Secure alarm to headboard and attach clip to patient's/resident's pajamas as shown in illustration B. Adjust cord to desired length.
4. Set alarm at high or low.



Important: To extend battery life, always place switch in "Off" position when alarm is not in use.

Battery Replacement

Skil-Care's Personal Alarm operates on a 9-volt battery. To access battery, push down on battery compartment slide on side of alarm. Remove battery and replace with a fresh one. Test alarm before use.