



**CORFLEX**

**CORFLEX IS A GLOBALLY BASED  
MANUFACTURER AND DISTRIBUTOR OF QUALITY  
ORTHOPEDIC AND REHABILITATION PRODUCTS**



**BACK MISC**

**MEDIC-AIR BACK PILLO**

- > Easily attaches to any chair to convert it into ergonomically correct seating; easily inflates to desired comfort level or deflates for portability
- > Provides support for the back and relief from lower back pain
- > Manufactured from medical grade nylon-coated vinyl with open-valve system



**PRODUCT NUMBER**

	<b>SINGLE</b>	<b>CASE/12</b>
Blue	84-2051-100	84-2051-300
Grey	84-2054-100	84-2054-300

**SIZING GUIDE**

Product measures 18" x 15"

**APPLICATION INSTRUCTIONS**

1. Unfold Pillo completely.
2. Gently & moderately blow air into Pillo with 3 or 4 breaths. (The Pillo should be soft rather than firm to gain the benefits of "floating air" action.)
3. Place cap firmly into valve (moisten slightly if it tends to stick.)
4. Hold Pillo on lap and with hand, press down in middle, forcing valve up. With the other hand, press on valve until it recedes into Pillo.

**CARE & CLEANING**

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Wipe exterior surface with damp cloth. You may use disinfectant or mild soap. Never use cleaning fluid, let pillow air dry, no not use any appliance for drying.

**HOW TO PLACE AN ORDER**

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



**TELEPHONE**

1-800-426-7353, Ext1



**FAX**

1-603-623-4111



**E-MAIL**

Service@corflex.com



**MAIL**

CORFLEX, Inc.  
Attn: Customer Service  
669 East Industrial Park Dr  
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109  
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.