



Finger Spring Proximal Interphalangeal Extension

Single Patient Use Only • Rx Only

IMPORTANT INFORMATION


The following information should be read prior to use of this device.

INTENDED USE

The Finger Spring fits from the head of the metacarpal to the distal interphalangeal joint. It is used to assist the proximal interphalangeal joint (PIP) into increased extension. It has a minimal effect on metacarpophalangeal (MP) extension.



Measuring for size: To determine splint length, measure involved finger as indicated in the diagram.

AA	2 1/4"	5.71 cm
A	2 5/8"	6.67 cm
B	3"	7.62 cm
C	3 3/8"	8.57 cm
D	3 3/4"	9.53 cm

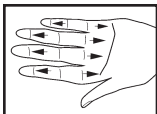

DIRECTIONS FOR USE


1. Select the correct size. To fit the spring, slide splint onto finger so the proximal pad rests over the volar aspect

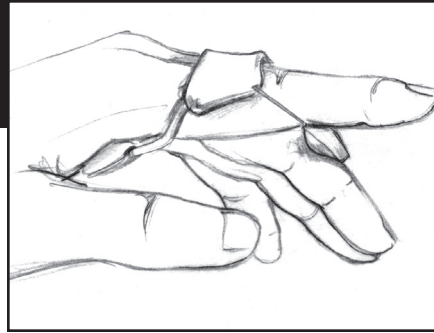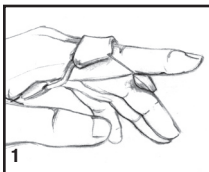


of the metacarpal head and the distal pad rests at the distal interphalangeal joint (DIP).

2. To adjust spring tension, there are two options:

a. To increase the force of extension, spread the volar pads apart before placing on the finger.

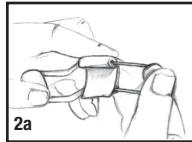

b. To decrease the force of extension, pinch the volar pads together before placing on the finger.

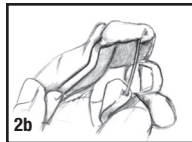

CARE INSTRUCTIONS

Splint can be cleaned by hand washing with a nonabrasive detergent. Air dry before wearing.



Manufacturer
DeRoyal
200 DeBusk Lane
Powell, TN 37849 U.S.A.
888.938.7828
865.938.7828

Part #1-507
©2009 DeRoyal. All Rights Reserved.
Revised 1/09



Finger Spring Proximal Interphalangeal Extension

Single Patient Use Only • Rx Only

IMPORTANT INFORMATION


The following information should be read prior to use of this device.

INTENDED USE

The Finger Spring fits from the head of the metacarpal to the distal interphalangeal joint. It is used to assist the proximal interphalangeal joint (PIP) into increased extension. It has a minimal effect on metacarpophalangeal (MP) extension.



Measuring for size: To determine splint length, measure involved finger as indicated in the diagram.

AA	2 1/4"	5.71 cm
A	2 5/8"	6.67 cm
B	3"	7.62 cm
C	3 3/8"	8.57 cm
D	3 3/4"	9.53 cm

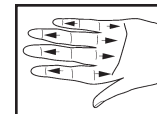

DIRECTIONS FOR USE


1. Select the correct size. To fit the spring, slide splint onto finger so the proximal pad rests over the volar aspect

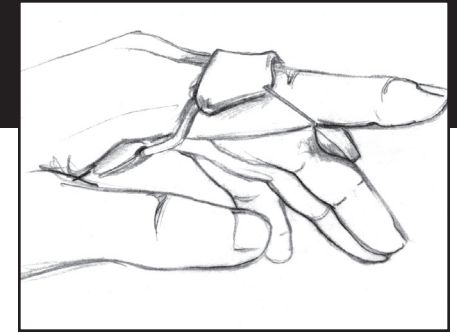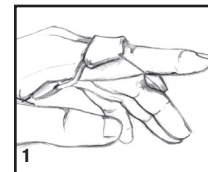

of the metacarpal head and the distal pad rests at the distal interphalangeal joint (DIP).

2. To adjust spring tension, there are two options:

a. To increase the force of extension, spread the volar pads apart before placing on the finger.

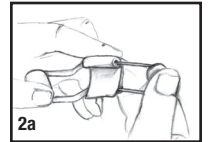

b. To decrease the force of extension, pinch the volar pads together before placing on the finger.



CARE INSTRUCTIONS

Splint can be cleaned by hand washing with a nonabrasive detergent. Air dry before wearing.



Manufacturer
DeRoyal
200 DeBusk Lane
Powell, TN 37849 U.S.A.
888.938.7828
865.938.7828

Part #1-507
©2009 DeRoyal. All Rights Reserved.
Revised 1/09