Manufacturer
DeRoyal
200 DeBusk Lane
Powell, TN 37849 U.S.A.
888.938.7828

Part #1-504

Revised (8/09)
©2001 DeRoyal. All Rights Reserved.

FINGER SPRING, PROXIMAL INTERPHALANGEAL EXTENSION
RESORTE PARA DEDO, EXTENSIÓN INTERFALANGIANA PROXIMAL
ATTELLE DE DOIGT À RESSORT POUR EXTENSION INTERPHALANGIENNE PROXIMALE
MOLA PARA DEDO, PARA EXTENSÃO DA INTERFALÂNGICA PROXIMAL

Part #1-504

Revised (8/09)
©2001 DeRoyal. All Rights Reserved.

FINGER SPRING, PROXIMAL INTERPHALANGEAL EXTENSION
RESORTE PARA DEDO, EXTENSIÓN INTERFALANGIANA PROXIMAL
ATTELLE DE DOIGT À RESSORT POUR EXTENSION INTERPHALANGIENNE PROXIMALE
MOLA PARA DEDO, PARA EXTENSÃO DA INTERFALÂNGICA PROXIMAL

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DIRECTIONS FOR USE
Fitting the Spring
1a. Hold the spring at the proximal and distal pads. **Note:** the proximal pad has the white plastic covered wire attached to it.
2a. Slide splint onto finger so the proximal pad rests over the volar surface of the proximal phalanx and the distal pad rests at the distal interphalangeal joint (DIP).
Adjust Spring Tension
1b. To increase the force of extension, spread the volar pads apart before placing on the finger.
2b. To decrease the force of extension, pinch the volar pads together before placing on the finger.
AA 1 3/4" 3 cm
A 1 7/8" 5 cm
B 2" 5,7 cm
C 2 1/4" 6,4 cm
D 2 5/8" 7,6 cm

INTENDED USE
This finger spring fits from the proximal phalanx to the distal interphalangeal (DIP) joint. It is used to assist the proximal interphalangeal joint (PIP) into increased extension.
To determine splint length, measure involved finger as indicated in the diagram.

IMPORTANT
Read completely before applying.
FINGER SPRING, PROXIMAL INTERPHALANGEAL EXTENSION

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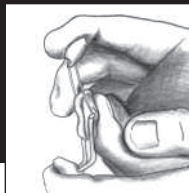
CARE INSTRUCTIONS
Splint can be cleaned by hand washing with a non-abrasive detergent. Air dry before wearing.
These instructions apply to product numbers: 504.
Single Patient Use

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