AirLITE® Cushion

Operations Manual

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Table Of Contents

Intended Use	2
Precautions	3
Parts Detail	4
Placement Instructions	5
Removing Your Cover	5
Cleaning & Disinfecting	6
Troubleshooting	6
Return & Contact Information	6
Warranty	7
Spanish	8
French	14
German	20
Italian	26
Dutch	32
Danish	38
Swedish	44
Norwegian	50
Finnish	56
Portuguese	62
Greek	68

ROHO, Inc. has a policy of continual product improvement and reserves the right to amend specifications presented in this manual.

ROHO® Cushioning Products are manufactured in the U.S.A. by ROHO, Inc.
100 N. Florida Avenue, Belleville, IL 62221-5429
U.S.A. & Canada: 800-851-3449, Fax 888-551-3449
Outside the U.S.A.: 618-277-9150, Fax 618-277-6518
www.therohogroup.com • cs@therohogroup.com

© 2003, 2007 ROHO, Inc.

The following are trademarks and registered trademarks of ROHO, Inc.: AirLITE®, AirLITE & design®, ROHO®, DRY FLOATATION®, and shape fitting technology®.

INTENDED USE: The AirLITE cushion is designed for users who weigh 250 lbs. (113 kg) or less, have partial to full sensation, are able to perform independent weight shifts, and require basic posture support and positioning needs to assist in the prevention of skin breakdown. The contoured foam pad has leg (femur) troughs and an ischial well that provide support for lower extremity and pelvic support and stability. The AirLITE cushion is intended to be used with the provided fluid-resistant cover.

CONTRA INDICATIONS: There are no known contra indications.

CLINICIAN CONSULTATION: ROHO recommends that a clinician (such as a doctor or therapist experienced in seating and positioning) be consulted to determine if the AirLITE cushion is appropriate for the user's particular seating needs.

- The air cell support pad is made of polyurethane (PU).
- The contoured foam pad is made of molded PU foam.
- The fluid resistant cover top is PU coated polyester. Its sides are PU coated nylon and the bottom is non-skid polyvinyl chloride (PVC).
- An optional, EN 1021-1 / EN 1021-2 compliant cover is available upon request. This cover's top is made of PU coated polyester with fire barrier sewn in. Its sides are PU coated nylon and the bottom is non-skid PVC.

Cushion	Width	Length
AL1515	15 in. (38 cm)	15 in. (38 cm)
AL1616	16 in. (41 cm)	16 in. (41 cm)
AL1618	16 in. (41 cm)	18 in. (46 cm)
AL1717	17 in. (43 cm)	17 in. (43 cm)
AL1816	18 in. (46 cm)	16 in. (41 cm)
AL1818	18 in. (46 cm)	18 in. (46 cm)
AL2018	20 in. (51 cm)	18 in. (46 cm)

Average Weight - 2 lbs. (1 kg)

Height ranges from 2 in. (5 cm) to 3.8 in. (10 cm). Range is due to contour in cushion.


NOTE: All measurements are approximate.

PRECAUTIONS

- **WEIGHT LIMIT:** DO NOT use if user weighs over 250 lbs (113 kg).
- **CUSHION AND COVER ORIENTATION:** Product must be used with the contoured foam facing up. If the cover is not used correctly or if the cover is the wrong size, it may reduce or eliminate the cushion's benefits and could increase risk to the skin and soft tissue.
- **DO NOT** expose your cushion to high heat, open flames, or hot ashes.
- **EXTREME COLD:** If a cushion has been in extreme cold (less than 32°F/0° C) and experiences unusual stiffness, allow the cushion to warm to 72°F/22°C before use.
- The AirLITE cushion is not a substitute for good skin care, including proper diet and regular pressure reliefs.
- **DO NOT** use on top of, or in conjunction with, another cushion.
- **OBSTRUCTIONS: DO NOT** place any obstructions between the user and the cushion as it will reduce product effectiveness.
- **DO NOT** use your cushion as a water flotation device (e.g. a Life Preserver).
- **OZONE GENERATORS:** Prolonged exposure to ozone will degrade materials used in your AirLITE cushion and may affect the performance of your product and invalidate the product warranty. Avoid prolonged exposure of the cushion to direct sunlight or other sources of ultraviolet (UV) light.

PARTS DETAIL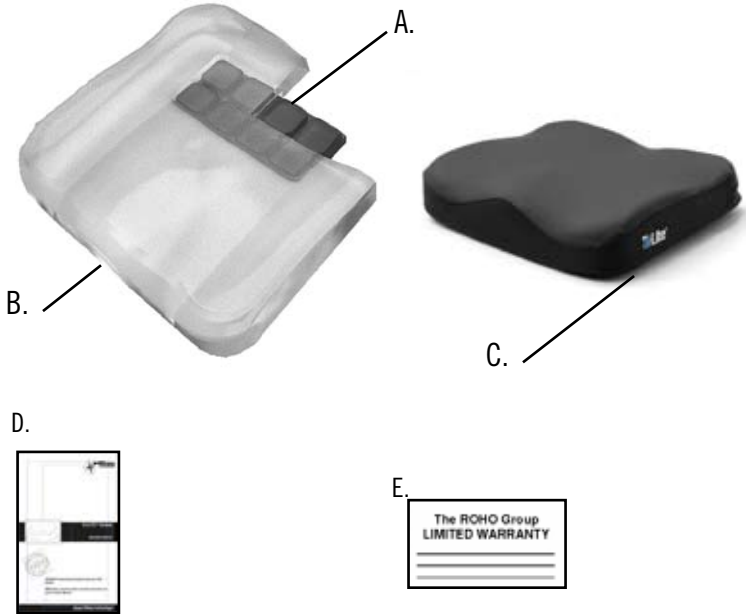

A. Support Pad: Individual interconnected air cells encased in foam.


B. Contoured Foam Pad: Encases the Support Pad.

C. Fluid Resistant Cover: Provides moisture protection to cushion.

D. Operations Manual: Your instructions on care, repairs, and other important information about your cushion.

E. Warranty Card: Complete warranty card and mail to ROHO or register online at www.therohogroup.com.


PLACEMENT INSTRUCTIONS FOR AIRLITE CUSHION:

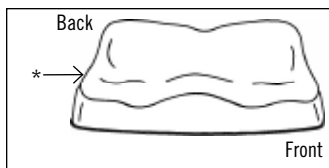* Patch is located on the front, right corner of the cushion.

STEP 1: Place cushion on chair, making sure it is centered. The directional patch indicates the front, back, top and bottom of the cushion.

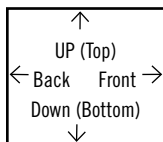STEP 2: Have the user sit in the chair, making sure the cushion is centered underneath.

CLEANING/DISINFECTING YOUR CUSHION:

Cleaning and disinfecting are two separate processes. Cleaning must precede disinfection.

CLEANING THE AIRLITE CUSHION

1. Remove cover.
2. Wipe gently with a cloth dampened with water and a household detergent.
3. Allow foam to air dry thoroughly.

DISINFECTING THE AIRLITE CUSHION

1. Cushion must be cleaned prior to disinfecting to remove any adhered soil.
2. Repeat cleaning instructions above using a household disinfectant instead of water and detergent.
3. Allow foam to air dry thoroughly.

REMOVING YOUR COVER:

Cover Removal:

1. Unzip cover and remove from cushion.

To cover your cushion, follow these simple instructions:

1. Unzip the cover and make sure the non-skid material is facing down.
2. Insert your cushion into the cover front first, with the contoured foam facing up. *Note: The label on the bottom on the cushion and the directional patch on the cover indicate proper orientation.*
3. Carefully close the zipper.

NOTES

1. **DO NOT** machine wash or dry the cushion.
2. **DO NOT** use cleaning products containing petroleum or organic solvents including acetone, toluene, MEK, naphtha, dry cleaning fluids, adhesive removers, etc.
3. **DO NOT** immerse cushion in water.
4. **DO NOT** autoclave, steam clean, or expose cushion to temperatures in excess of 150°F (65°C).
5. **DO NOT** expose cushion to ultraviolet light or ozone gas cleaning methods.

CLEANING/DISINFECTING YOUR COVER:

Cleaning and disinfecting are two separate processes. Cleaning must precede disinfection.

CLEANING THE AIRLITE CUSHION COVER

1. Remove cover.
2. Machine wash in cold water (30° C) with mild detergent, gentle cycle. No bleach.
3. Air dry.

DISINFECTING THE AIRLITE CUSHION COVER

1. Machine wash in hot water (60° C) with detergent.
2. Air dry.

TROUBLESHOOTING:

Uncomfortable/Unstable: Make sure the cushion is the correct size and properly oriented. Allow the user to sit on the cushion for at least one hour to get used to the texture of the product's surface and immersion into the cushion.

Cushion Slides On Chair: Make sure non-skid bottom of the cushion cover is facing "DOWN".

Return & Contact Information:

All returns require prior authorization from ROHO, Inc. and are subject to a restocking charge.

Before returning your product to ROHO Inc., contact our Customer Support Department at 1-800-851-3449 toll free in the **U.S.A.** for a Return Authorization Number. A Return Authorization Form must be included with your product when returned, which can be obtained from our website at www.therohogroup.com or by contacting Customer Support. **Outside the U.S.A.**, contact your country's distributor or contact ROHO International at 1-618-277-9150 for your nearest distributor. Check our web site, www.therohogroup.com, for the latest list of ROHO International distributors. You may also e-mail us at: cs@therohogroup.com.

Disposal

When properly used and disposed, there are no known environmental hazards associated with the components of the AirLITE cushion. Dispose of the product and / or components in accordance with the applicable regulations in your jurisdiction.

Limited Warranty

What is covered?

ROHO, Inc. covers any defects in materials or workmanship of the product. Any substitutions of supplied components will void the warranty.

For how long?

24 months from the date the product was originally purchased, with the exception of the cover, which has a 12-Month Limited Warranty.

What we will do.

Within a reasonable amount of time after you return the product to us, we will repair or replace any defect in material and workmanship and ship it back to you, all free of charge.

What we will not do.

We will not repair a product free of charge if it has been misused, damaged by an accident, or damaged from "an act of Nature", e.g., flood, tornado, earthquake, fire.

What the customer must do.

Within the **U.S.A.**, the customer must contact ROHO, Inc. to obtain a Return Authorization Number and a completed Return Authorization Form must be included with the product when returned. The Return Authorization Form can be obtained from our website at www.therohogroup.com or by contacting Customer Support at 1-800-851-3449 toll free.

Outside the U.S.A., contact the nearest ROHO distributor. See website for current list of ROHO distributors (www.therohogroup.com).

Failure to submit a Return Authorization Form may result in the product being returned without evaluation or repair.

Questions.

Call our Customer Support Department toll free from **U.S.A.** at (800)-851-3449. **Outside the U.S.A.** call your distributor or the ROHO Customer Support Department at 1-618-277-9150.

Disclaimer

Any implied warranties, including the MERCHANTABILITY and FITNESS FOR A PARTICULAR PURPOSE are also limited to the period of time specified above from the date of original purchase and any action for a breach of such warranties or any express warranties herein must be commenced within this time. ROHO, Inc. shall not be responsible for any incidental or consequential damages with respect to economic loss or injuries to property, whether as a result of a breach of express or implied warranties, or otherwise. ROHO, Inc. will not be responsible for the results of careless handling, unreasonable or improper use of this product.

Your Rights and the Effect of this Warranty upon Them

This warranty restricts the duration of all implied warranties as noted above and limits or excludes recovery for incidental or consequential damages. Some jurisdictions do not allow limitations on how long implied warranties will last and some jurisdictions do not allow the exclusion or limitation of consequential or incidental damages, so those provisions may not apply to you. While this warranty gives you specific legal rights, you may also have other rights which vary from jurisdiction to jurisdiction.

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MDSS GmbH
of ROHO, Inc. are certified to
Belleville, IL 62221-5429 U.S.A.