

# COMPRESHORTS



## STANDARD SIZE

### FEATURES

10–15mmHg of supportive, mild compression for truncal / abdominal lymphedema

Soft, breathable fabric reduces moisture containment

Reinforced seams for durability

Machine wash and dry

May be paired with the GENIFIT CHIP PAD for additional genital support (see page 53)

*Elastic band on sizes 2XLRG and above contains latex*

### INDICATIONS

Stage I or II Lymphedema

Mild fibrosis


Post-surgical edema

### INCLUDES

1 COMPRESHORTS garment

COMPRESHORTS SIZING CHART (WAIST, HIP, MIDTHIGH)

	WAIST	HIP	MIDTHIGH	ITEM NUMBER
<b>SMALL</b>	69–76cm	84–91cm	41–46cm	<b>1301-CS</b>
<b>SMALL MAX</b>	76–86cm	91–102cm	41–46cm	<b>1301-CM</b>
<b>MEDIUM</b>	76–86cm	91–102cm	46–53cm	<b>1302-CS</b>
<b>MED. MAX</b>	86–97cm	102–112cm	46–53cm	<b>1302-CM</b>
<b>LARGE</b>	86–97cm	102–112cm	53–61cm	<b>1303-CS</b>
<b>LRG MAX</b>	97–112cm	112–127cm	53–61cm	<b>1303-CM</b>
<b>XLARGE</b>	97–112cm	112–127cm	61–71cm	<b>1304-CS</b>
<b>XLRG MAX</b>	112–132cm	127–147cm	61–71cm	<b>1304-CM</b>
<b>2XLARGE</b>	112–132cm	127–147cm	71–81cm	<b>1305-CS</b>
<b>2XLRG MAX</b>	132–152cm	147–168cm	71–81cm	<b>1305-CM</b>
<b>3XLARGE</b>	132–152cm	147–168cm	81–91cm	<b>1306-CS</b>
<b>3XLRG MAX</b>	152–172cm	168–188cm	81–91cm	<b>1306-CM</b>
<b>4XLARGE</b>	152–172cm	168–188cm	91–101cm	<b>1307-CS</b>
<b>4XLRG MAX</b>	172–193cm	188–208cm	91–101cm	<b>1307-CM</b>
<b>5XLARGE</b>	172–193cm	188–208cm	101–112cm	<b>1308-CS</b>

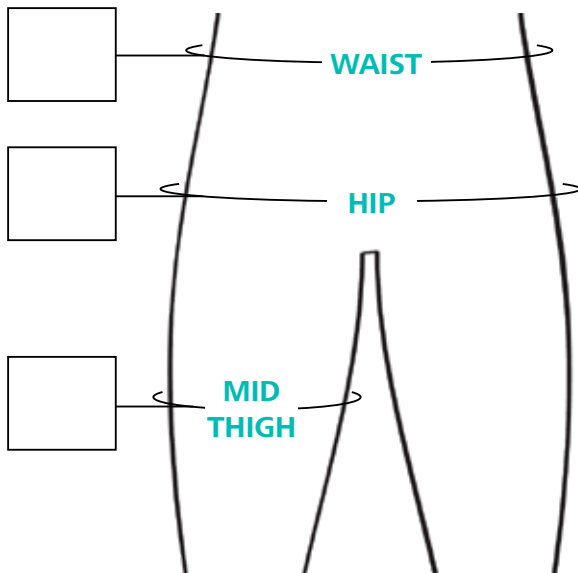

P.O. #	Account #	Contact	Date
Phone	Patient Name	Age	Height Weight
Company Name			
Dropship Name & Address			
Product Information			
Size <input type="checkbox"/> STANDARD	Size <input type="checkbox"/> CUSTOM		
Standard Size	Custom Waist Measurement		
Item #	Custom Hip Measurement		
	Custom Midthigh Measurement		

A PDF of this order form can be found online at [bit.ly/SIGmccOrderforms](http://bit.ly/SIGmccOrderforms)

## CUSTOM SIZE

Excellent choice for patients with lipedema, or who are outside of standard size ranges. Custom length available.

### STEP ONE

Record the **WAIST**, **HIP**, and **MIDTHIGH** measurements

### STEP TWO

Measure from the groin down and indicate the garment inseam length, record in the corresponding box

### STEP THREE

Mark the end of the inseam and take a circumference measurement at that point, record in the corresponding box

After recording these measurements, please call Customer Care: 1.866.931.0876.

