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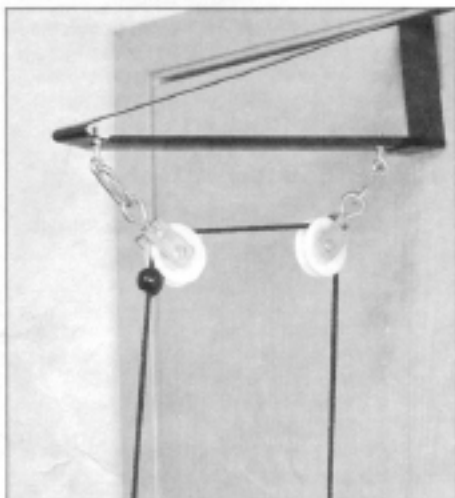
D

E

Distributed by:
Performance Health Supply Inc.
28100 Torch Parkway, Ste. 700
Warrenville, IL 60555, USA
Phone: 800-323-5547
Fax: 800-547-4333

Manufacturer:
Performance Health Supply Inc.
W68 N158 Evergreen Blvd.
Cedarburg, WI 53012, USA
Phone: 800-228-3693

Distributed by:
Performance Health Canada, Inc.
6675 Millcreek Drive, Unit 3
Mississauga, ON L5N 5M4
Phone: 800-665-9200
Fax: 877-566-8611

Authorized European
Representative
**Performance Health
International, LTD.**
Nunn Brook Road
Huthwaite, Sutton-in-Ashfield
Nottinghamshire NG17 2HU, UK
Phone: +44 (0) 1623 448 706
Fax: +44 (0) 1623 448 784

Rolyan* Reach 'N Range* Pulley:

A873-623 and A873-624

Rolyan* Reach 'N Range* Pulley with Assist:

A873-621 and A873-622

Rolyan* Reach 'N Range* Overhead Pulley:

A873-620

Be sure to note which product you are using and follow the instructions that apply to that product. Unless otherwise stated, these instructions apply to all of the products listed.

Caution: Not designed to support body weight. To be used in a sitting or kneeling position. Inspect before each use.

INDICATIONS

- To increase upper extremity range of motion and strength
- May be used following shoulder injury, CVA, and peripheral nerve injury

CONTRAINDICATIONS

- Not for use in the presence of fractures
- Not for use in the presence of severe edema

PRECAUTIONS

- If unusual swelling, skin discoloration or discomfort occurs, use should be discontinued and a healthcare professional consulted.
- Door supporting pulley should be closed firmly and locked prior to pulley use.
- An individual with an acute cardiac condition should seek advice of a healthcare professional prior to using this device.

Rolyan Reach 'N Range Pulley with Assist

- Be careful that foam handle is not so tight that circulation is impaired.

CARE AND CLEANING

Wipe with warm water and mild soap and allow to dry thoroughly.

COMPONENTS

The three pulley systems include the following components:

Rolyan Reach 'N Range Pulley

- 2 padded handles
- either webbing strap or metal bracket for attachment to door

Rolyan Reach 'N Range Pulley with Assist

- 1 padded handle
- 1 foam handle that supports extremity without the need to grip a handle
- either webbing strap or metal bracket for attachment to door

Rolyan Reach 'N Range Overhead Pulley

- 2 padded handles
- 2 pulleys
- attachment bracket that extends out from door

GENERAL INSTRUCTIONS

1. Use standard-width door.
2. Attach bracket as follows:
Metal bracket: With door open, place bracket onto top of door and push down firmly (A). Close door and lock it. Unfold pulley cord. Pull on unit to be sure bracket is properly in place and door will not open unexpectedly.

Webbing strap: With door open, place strap over door (B). Close door firmly and lock it. Unfold pulley cord. Pull on unit to be sure slack on strap is taken up and door will not open unexpectedly.

Dual pulley attachment bracket: With door open, place bracket onto top of door and push down firmly onto door (C). Close door and lock it. Unfold pulley cord. Pull on unit to be sure bracket is properly in place and door will not open unexpectedly.

3. Adjust length of pulley line as follows:

Padded handle: To shorten, pull unwanted length through holes on plastic adjustment piece and handle. Tie knot. Test cord to be sure knot is tight before each use of pulley.
Soft Foam handle: To shorten, pull unwanted length through grommet in handle and tie knot. Test cord to be sure knot is tight before each use of pulley.

4. Use ROM stops on pulley cord to adjust allowed range of motion. Slide cord through hole in bead to raise or lower bead on cord (D). Tighten cord around bead.
5. If using padded handles, grasp them as shown in B. If using soft foam handle to support extremity, place hand through foam loop and support wrist as shown in E. Tighten strap as needed around wrist.

Figures referred to in the instructions are on page 4.

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