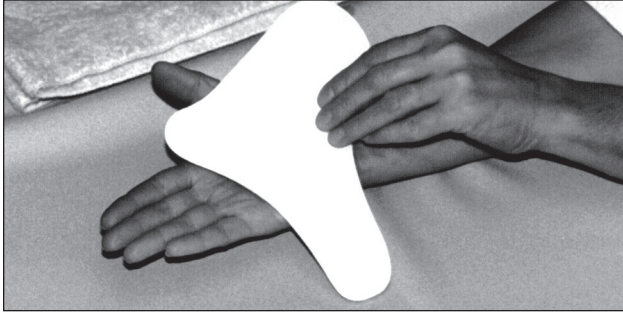
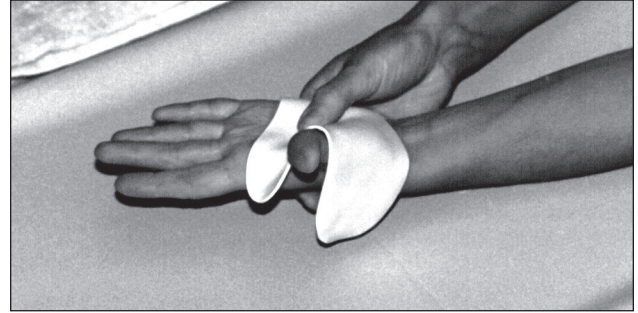
Rolyan® Hand-Based Thumb Spica Splint:

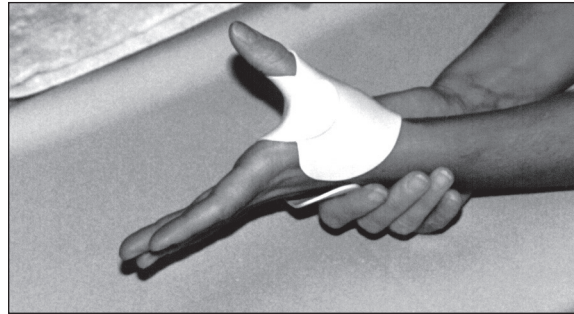
A358-1 and A358-2, 5504-11,-12,-16,-17, and 5514-31

A

B

C

INDICATIONS

- For immobilizing the CMC and MCP joints of the thumb
- May be used for cumulative trauma injuries, collateral ligament strain, acute arthritis, median nerve injury, and tendonitis

CONTRAINDICATIONS

- Not in the presence of severe edema
- Not for use directly over open wounds and infected areas
- May not be recommended for the acute phase of most thumb fractures

PRECAUTIONS

- **This splint is to be fitted initially by a healthcare professional who is familiar with the purpose for which it is prescribed. The healthcare professional is responsible for providing wearing instructions and precautions to other healthcare practitioners, care providers involved in the patient's care, and the patient.**
- **If unusual swelling, skin discoloration or discomfort occurs, use should be discontinued and a healthcare professional consulted.**
- **Be careful that the straps are not so tight that they interfere with circulation.**
- **After heating splinting material, always check its temperature before applying it to the patient.**

CARE AND CLEANING

- The splint will lose its shape in temperatures over 135°F (57°C). It should be kept away from sources of heat such as ovens, hot water, open flames, and sunny car windows.
- The splint may be cleaned with soap and lukewarm water. The straps may be washed with soap and water, but the adhesive portion should not be moistened. The splint and straps should be allowed to dry thoroughly before reapplication of the splint.

PROCEDURE

A splint blank that is not pre-padded can be applied to either hand—simply invert the splint blank.

The splint can be applied to the volar or dorsal surface of the hand. If the splint blank is pre-padded, use a left splint for a dorsal splint on the right hand and a right splint for a dorsal splint on the left hand.

The instructions and photographs describe the splint applied volarly. For dorsal application, position the splint blank in the same position—with the longest corner extending past the ulna and the shortest corner at the thumb web space (A).

1. Heat splint blank in water heated to approximately 160°F (70°C) until pliable. (Refer to the instructions for Rolyan splinting materials for information about working with specific splinting materials.)

2. Remove from heat pan and place on a towel on a flat surface. Blot excess water.
3. If possible, place patient's hand in supination to achieve a gravity-assisted position. Place splint blank on the volar surface as shown in A. (For dorsal application, place the patient's hand in pronation and place the splint blank on the dorsal surface.)
4. Mold the splint blank around the thumb (B), overlapping the material (C).
5. Mold the ulnar border of the splint over the dorsum of the hand (C).
6. The thumb piece can be left free to accommodate fluctuation in edema or bonded to the splint base as follows:
 - If the splinting material is coated to prevent accidental bonding, remove the coating by scraping the surface or brushing it with an acetone material.
 - The best bond can be achieved by dry heating the material (such as with a heat gun).
 - While still warm, place the pieces together carefully—they will bond instantly.
7. The splint can be trimmed and the edges flared if necessary.
8. Secure a volar splint with a dorsal strap; secure a dorsal splint with a palmar strap.

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Patterson Medical, Ltd.