

first class chair footrest

item# FC 2028 & FC 4028
(for use with item# FC 2000 & FC 4000)



EU Authorized Representative
Drive International, LLC
Globe Lane - Stockport
DUKINFIELD SK16 4RE, GB

Wenzel **Yte** *Re/hab*

drive
WWW.DRIVEMEDICAL.COM

installation



1. Using the allen wrench provided, remove hex screws (A) and separate mounting brackets (Figure 1).
2. Mount brackets on front legs.
NOTE: Position opening with plastic collar (B) around the leg.
Do not tighten hex screws.
3. Insert chrome footrest tube into mounting bracket and place at desired position (Figure 2).
4. Tighten hex screws.

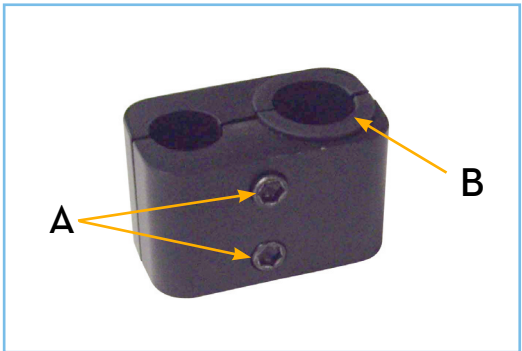


figure 1

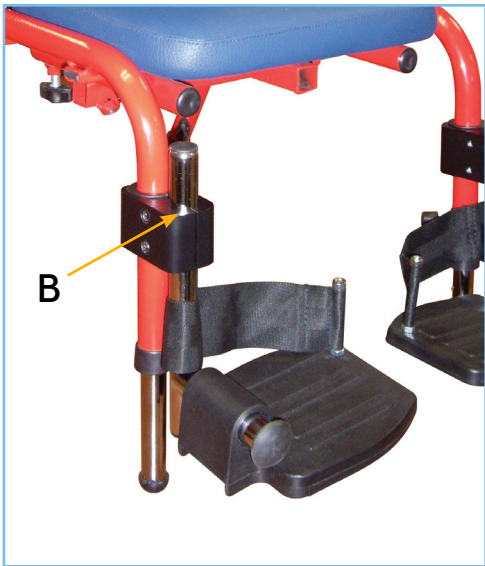


figure 2

two year limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for two years from the date of the original consumer purchase.

This device was built to exacting standards and carefully inspected prior to shipment. This Two Year Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

drive
WWW.DRIVEMEDICAL.COM

Wenzel *ite* *Re/hab*

© 2006 Medical Depot, Inc. All rights reserved.

Drive is a trademark of Medical Depot, Inc.

Port Washington N.Y. 11050 USA Made in China.