

912410



912416

ReOrder #	Description	Wheelchair Size	Unit
912416	Gel-Top Velcro® Closure ( <i>Pediatrics</i> )	14"	Each
912410	Gel-Top Lift-Away/Restraint Free	16" - 18"	Each
912415	Gel-Top Velcro® Closure	16" - 18"	Each

### Purpose


Skil-Care's Gel-Top Sensory Stimulator Tray provides both tactile and visual stimulation to reduce agitation and restlessness on the part of wheelchair-bound Alzheimer's and other diminished capacity patients. It also reduces pressure on arms and elbows when the patient uses the tray for postural support and positioning.

### Easy to Use

The Gel-Top Sensory Stimulator Tray is available in two models: A Lift-Away version for restraint-free care and a Velcro® Closure version.

#### **Lift-Away Model (#912410)**

1. Wrap padded armrest covers around armrests and secure Velcro® (Illustration A)  
Make certain that the small straps attached to the armrest covers are positioned to the rear of the armrests.
2. Wrap the small strap around the armrest support bar, just behind the wheelchair armrest

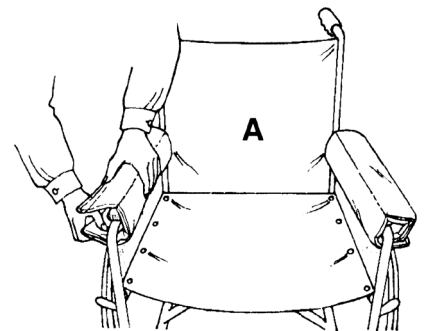

#### **Velcro® Closure Model (#912415 & 912416)**

1. Place tray on wheelchair arms. Wrap the Velcro® bands around chair arms and tray. Secure the bands by pressing the "hook" material.

### Easy to Care

All trays may be cleaned with sprayed-type cleanser and a damp cloth or sponge. The armrest covers can be laundered and dried at temperatures up to 180°F (82°C). Do not use bleach.

