

Gel Knee Protector Button 562056

Installation Instructions:

The gel knee protector buttons are designed to go at the top, front pivot point of most elevating leg rests.

Remove existing nut and bolt from the pivot joint.

Replace the bolt with the 2" x 1/4" bolt provided. If the removed bolt is larger than the 1/4", use the steel spacer the size of the removed bolt over the bolt before reinserting in to the pivot joint.

Attach the 1/4" nylock nut and tighten until firm. Do not over tighten.

Install plastic back cover over the bolt, smooth side away from the leg rest and towards the gel knee button.

Screw the gel knee protector button onto the end of the bolt.

Cleaning Instructions:

The exterior surface of the gel pad can be washed with any commercial detergent by scrubbing and rinsing with a wet sponge or cloth. Do not submerge gel pad in water or put in washing machine as gel may dissolve.



Call toll free: 1-800-323-5547

In Canada 1-800-665-9200

Fax toll free: 1-800-547-4333

In Canada 1-905-566-9207