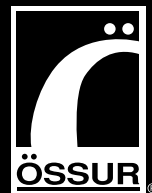
Instructions for use

# Innovator® Premium Model

*Life Without Limitations®*

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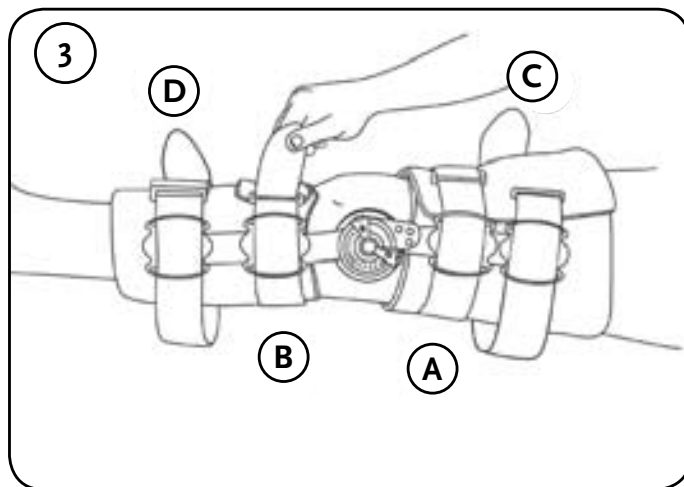
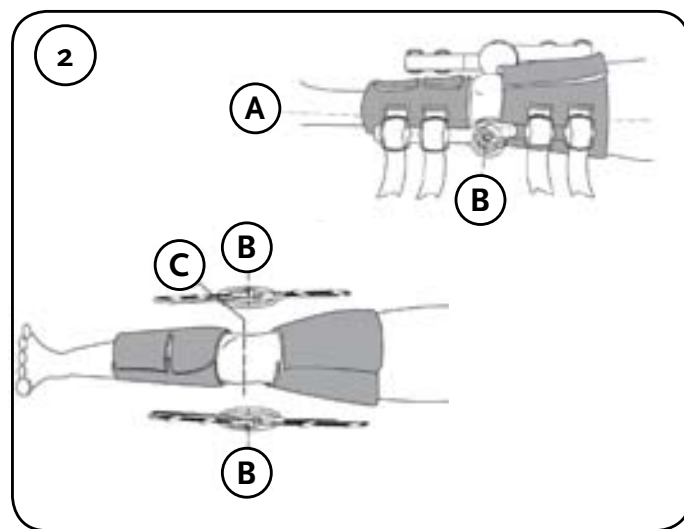
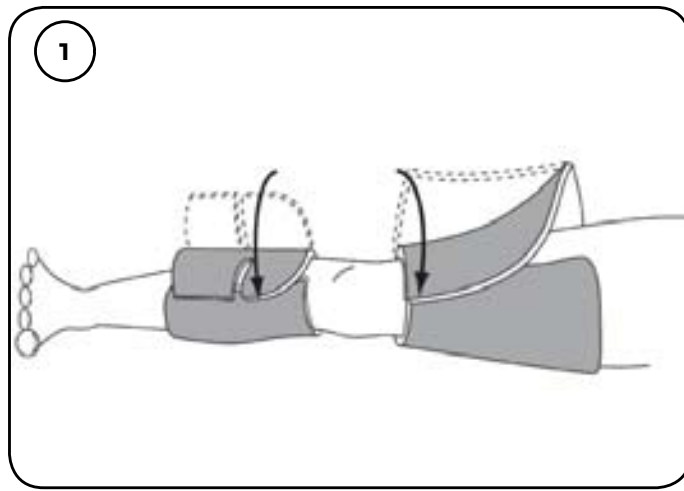
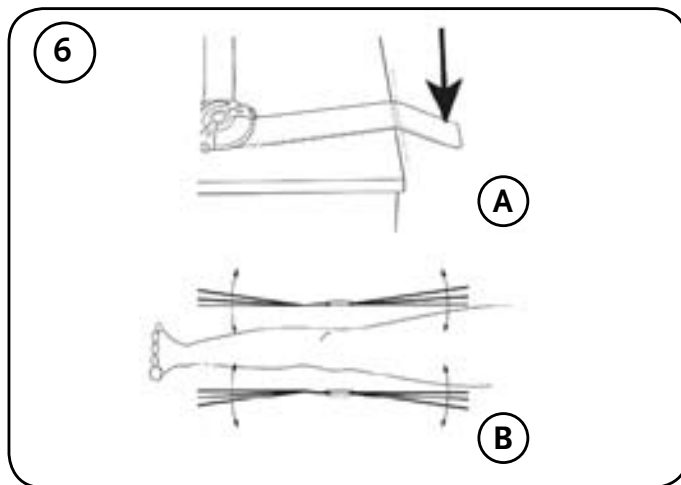
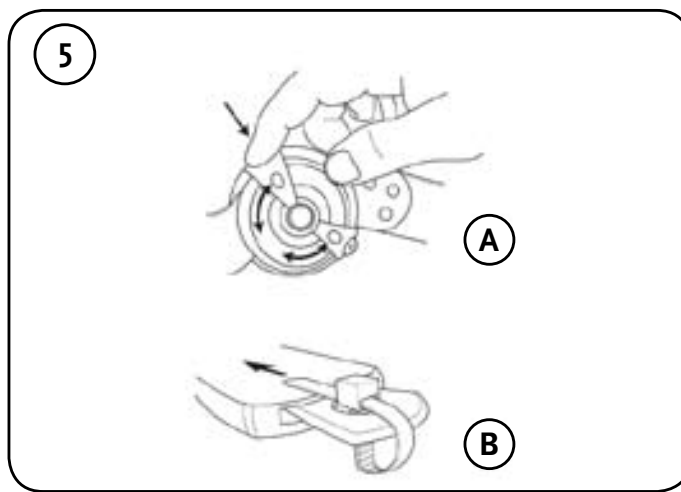
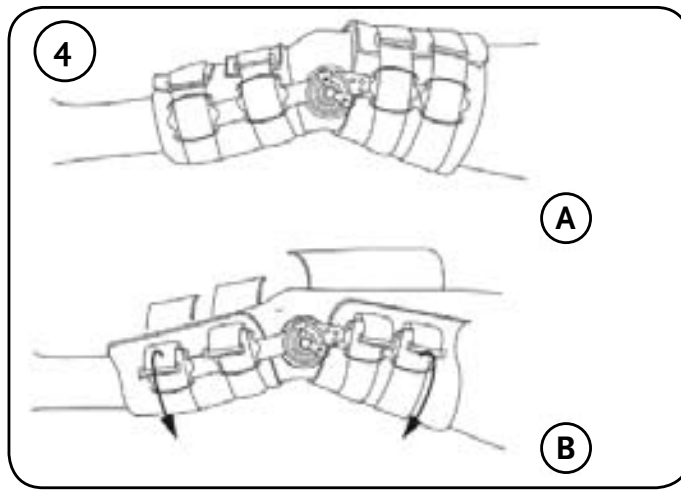
---

## APPLY THE FOAM WRAPS

1. Remove foam cuffs from hinge assemblies
2. Apply calf and thigh foam cuffs snugly around the leg (Figure 1) closing on front of the leg.

## ALIGN AND APPLY HINGES

1. Position and press the lateral side bar assembly onto the cuffs along the midline of the leg (Figure 2a), centering the hinge pivot axes (Figure 2b) slightly above the knee joint line (Figure 2c).
2. Repeat for the medial side bar assembly, ensuring that the hinge pivot axis (Figure 2b) align vertically.
3. Reposition the foam cuffs as needed, ensuring that the patella remains exposed.

## SECURE THE LEG STRAPS

1. Secure and fasten all straps, starting with the straps closest to the knee first (Figure 3a,b,c,d).

## BRACE APPLICATION AND REMOVAL

### A properly applied brace (Figure 4a) will have:

1. Foam wraps secured at the front of the leg,
2. Hinges will be aligned vertically with each other at the midline of the leg, and pivot just above the knee axis,
3. All straps will be secured at the front of the leg.

### Removal and Re-application

4. To remove the brace, unfasten the straps and open the foam pads (Figure 4b). The brace can then be removed and reapplied as a single unit.

## HINGE OPERATION AND ADJUSTMENT

1. To adjust the hinge, press the flexion or extension button and rotate until the desired degree of flexion or extension shows through the corresponding window and release button (Figure 5a). **The “o-lock” settings are used to lock the brace in full extension.**

### Enforced Compliance

2. Loop the enclosed lock ties through the hole in the flexion and extension buttons to limit range of motion (Figure 5b).

## SIDE BAR ADJUSTMENT

1. To breakdown the side bars, place the hinge on a hard flat surface with the scored line just off the edge and firmly press down to snap off (Figure 6a).
2. Side bars can be bent to fit individual legs (Figure 6b).

## USE AND CARE

Read all instructions completely before use.  
All foam pads may be trimmed as necessary.  
Hand wash foam liners with mild soap.  
Air dry only. Do not heat dry.

**Always consult your physician or therapist before making any adjustments to the brace.**

## CAUTIONS AND WARNINGS:

1. Carefully read use instructions and warnings prior to operation.
2. Federal law restricts this device to sale by or on the order of a licensed health care practitioner.
3. For single patient use only.
4. If you experience increased pain, swelling, sensation changes or any adverse reactions while using this product, immediately consult your medical professional.
5. This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also essential parts of a safe treatment program.

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Innovator ® Premium is covered by one or more of the following patents. Other U.S. and foreign patents pending. ® indicates trademark registration in U.S.A. and selected countries only.