AUSTRIA

Bauerfeind Ges.m.b.H.
Hainburger Straße 33
1030 Wien
P +43 (0) 800 4430-130
F +43 (0) 800 4430-131
E info@bauerfeind.at

BELUX

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 english

These instructions for use can also be downloaded from www.bauerfeind.com/downloads.

Dear Customer,

Thank you for placing your trust in a Bauerfeind product. In choosing the CaligaLoc, you have acquired a high-quality medical product.

Please read through all of these **instructions for use** carefully, because they contain important information about the use, indications, contraindications, fitting, care and disposal of the CaligaLoc. Please keep these instructions in case you need to refer to them again in the future. If you have any questions, please contact your physician or medical retailer.

Intended purpose

The CaligaLoc is an anatomically contoured orthosis¹ for the ankle joint. The CaligaLoc prevents the lateral twisting of the ankle to the outside and provides effective support for the muscular stabilization of the upper and lower ankle. The outside of the foot is raised, providing additional relief for the outer ligaments. The CaligaLoc can be adjusted individually using the adjustable Velcro straps. CaligaLoc is suitable for use in comfortable outdoor and sports shoes, and can also be worn without shoes overnight.

Indications

- Non-surgical treatment of torn ankle ligaments or a badly sprained ankle joint (distortion)
- Protection following reconstructive ligament surgery
- Permanent stabilization for complex instability of the upper and/or lower ankle

Risks of using this product

Caution²

- CaligaLoc must only be worn in accordance with the specifications contained in these instructions for use and the areas of application (places of use) listed. To ensure the optimum fit of the CaligaLoc, it is essential to check whether the product is adequately shaped. If necessary, the CaligaLoc can be anatomically re-shaped by a specialist³.
- Please consult your physician in advance if you intend to use this product in combination with other products, e.g. if you wear compression stockings for compression therapy.
- Should you notice any unusual changes (e.g. increased symptoms), please contact your physician without delay.
- No product liability is accepted in the event of improper use.
- Side effects involving the body as a whole have not been reported to date. Correct usage and fitting is assumed. Do not fit the CaligaLoc too tightly, as this could lead to local pressure symptoms. In rare cases, constriction of blood vessels and nerves may occur. If this happens, loosen the straps of the CaligaLoc and, if necessary, check the size of your CaligaLoc.

Contraindications

Hypersensitivity reactions harmful to health have not been reported to date. In the following conditions such aids should only be fitted and applied after consultation with your physician:

- Skin disorders/injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up.
- Impaired sensation and movement disorders of the legs/feet, e.g. in the case of diabetes mellitus
- Impaired lymph drainage, including soft tissue swellings of uncertain origin located remotely from the fitted orthosis

Application instructions

Putting on/fitting the CaligaLoc

A – plastic shell, B – upper strap,
C – lower strap, D – middle Y-shaped strap

We recommend that hosiery should be worn under the CaligaLoc.

Sit down on a chair to put on the CaligaLoc. Place your foot on the floor with your foot at a 90° angle to your lower leg, as this is the best position for fitting the orthosis.

- 1 Undo all the straps. Place the relevant foot on the sole of the CaligaLoc, taking care to ensure the correct position of the heel. The outside of the back of the foot should now be encased by the plastic shell.
- 2 Now wrap the upper strap round your lower leg from behind and guide it through the loop in the shin bone area.
- 3 Secure the end of the strap by fastening the Velcro to the section of strap which is underneath it.
- 4 Guide the lower strap over the instep of the foot to the outside of the foot and secure it in place with the Velcro. If necessary, the strap can be cut with scissors to make it shorter.
- 5 For the Y-shaped strap, proceed as follows: Guide the front strap in front of the shin bone and
- 6 the rear strap round the heel into the guide loops, then use the Velcro to secure them to the section of strap underneath. Tighten the straps again if necessary.

Taking off/removing the CaligaLoc

Undo all the Velcro straps and lift your foot up and forward out of the orthosis.

Cleaning instructions

Note: Never expose the CaligaLoc to direct heat (e.g. heater, sunlight, in your car, etc.)! This could damage the material. This can impair the effectiveness of the CaligaLoc.

The sewn-in label of the CaligaLoc, which contains information on the product name, size, manufacturer, washing instructions and CE certification, is located underneath the upper strap of the orthosis.

First undo all of the Velcro fastenings. Then pull each strap out of its loop completely. Fasten the ends back onto the straps to avoid any damage. Use a damp cloth to wipe the shell and cushion. Use a mild cleanser.

Permissible conditions and places of use

In accordance with the indications (ankle joint).

Maintenance instructions

With correct handling and care, the product is practically maintenance-free.

Assembly instructions

The CaligaLoc is supplied ready for use.

Technical data/parameters

The CaligaLoc is an ankle orthosis with three attached straps. The straps are not removable.

Notes on reuse

The CaligaLoc is intended for your sole use and has been individually adapted for you. For this reason, do not pass the CaligaLoc on to other people.

Warranty

The legal provisions of the country in which the product was purchased apply. Country-specific warranty regulations that govern the relationship between the retailer and the buyer also apply where relevant. If you believe that a claim may be made under the warranty, please first contact the retailer from which you purchased the product directly.

Please do not make any changes to the product yourself. This particularly applies to individual adjustments made by specialists. Please follow our instructions for use and care. These are based on our experience gained over many years and ensure the functionality of our medically effective products for a long time. Only aids that function optimally can help you. Failure to follow the instructions may also restrict the warranty.

Disposal

You can dispose of the CaligaLoc in accordance with national legal requirements.

Version: 2017-03

- 1 Orthosis = orthopedic aid used to stabilize, relieve, immobilize, control or correct the limbs or the torso
- 2 Note on the risk of personal injury (risk of injury, harm to health and accidents) or damage to property (damage to the product).
- 3 A specialist is any person who is authorized according to the state regulations for fitting and instruction in the use of supports and orthoses which are relevant to you.

Specialists

General notes

- Check the optimal and individual fit of the CaligaLoc at the first fitting. Make any corrections if necessary.
- Practice the correct procedure for putting on the orthosis with the patient.
- The product must not be changed in any improper way. Failure to comply with this requirement may adversely affect the medical and therapeutic effectiveness of the product, thereby voiding any product liability.
- The CaligaLoc is intended for the treatment of one patient only.

Adapting the CaligaLoc

Adjusting the plastic shell

- 1 Check the fit of the CaligaLoc on the patient. Pay careful attention to the patient's individual anatomy (e.g. the lateral malleolus).
- 2 Carefully remove the cushion from the plastic shell.
- 3 If necessary, adapt the CaligaLoc using a hot-air gun at 120-140 °C (250-280 °F).
- 4 Wear protective gloves when you are doing this.
- 5 Glue the inner cushion back into the plastic shell.

Adjusting the cushion

- 6 If the ankle is badly swollen and the cushion is therefore pressing against the lateral malleolus, we recommend making a cutout in this part of the cushion. Use sharp scissors or a cutter knife and cut along the embossing.

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