AUSTRIA

Bauerfeind Ges.m.b.H.
Hainburger Straße 33
1030 Wien
P +43 (0) 800 4430-130
F +43 (0) 800 4430-131
E info@bauerfeind.at

BELUX

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en english

These instructions for use can also be downloaded from www.bauerfeind.com/downloads.

Dear Valued Customer,

Thank you for choosing a product from Bauerfeind. EpiTrain is a high-quality medical product. If there is a CE mark on the label sewn into your support, this means it is a standard product. If your name is printed on the sewn-in label and there is no CE mark, this means it is a made-to-measure product specially manufactured in accordance with Section 3 No. 8 of the German Medical Devices Act. Please carefully read through the following instructions for use. Should you have any questions contact your medical provider, medical retailer or our customer service department.

Purpose

EpiTrain is particularly effective during movement and exercise treatment. Intermittent compression on the soft tissue parts of the joint is achieved by the knitted fabric and the integral pads made from a viscoelastic material. The improved flow of blood through the joint capsule and the promotion of lymph drainage lead to reduced swelling, pain relief, and improved function. A strap is attached to increase the pain-relieving effect. This can be tightened or loosened or completely removed as needed.

⚠ EpiTrain must be fitted for initial use by a trained specialist¹, otherwise we cannot accept any liability for the product.

Indications

- Irritated tendon – e.g., tennis elbow, golfer's elbow (epicondylopathies)
- Worn / inflamed joint (arthrosis / arthritis)
- Post-operative and post-traumatic irritation
- Muscle and tendon disorders (tendomyopathies)
- Prevention / prophylaxis against relapses

Risks of using this product

⚠ Important Information

- EpiTrain is a prescribed product that should be applied under a physician's guidance. EpiTrain should only be applied in accordance with these instructions for use and for the specified areas of application.
- No liability is accepted for improper use of the product.
- Changes must not be made to the product. Non compliance can reduce the performance of the product, thus no liability will be accepted.
- Should you notice any unusual changes (e.g., increased symptoms), please contact your medical provider without delay.
- A combination with other products must be discussed with your medical provider prior to use.
- The product's unique benefits are demonstrated primarily during physical activity. The product should be removed during prolonged periods of rest (e.g., sleeping).
- Avoid the use of ointments, lotions or any substances that include grease or acids.
- With the assumption of proper fitting, no side effects involving the body as a whole have been reported to date. Supports and orthoses² applied externally to the body can, if tightened excessively, lead to localized pressure or, rarely, constrict the underlying blood vessels or nerves. If fastened too tightly, the optional strap can lead to bonding of the fasciae in cases of acute inflammatory changes and therefore restriction of movement and pain.

⚠ Contains natural rubber latex (Elastodiene) which can cause skin irritation in persons with an allergy. Take off the support if you notice any allergic reaction.

Contraindications

Hypersensitivity reactions harmful to health have not been reported to date. For patients experiencing

the following conditions, such supports and orthoses should only be fitted and applied after consultation with your medical provider:

1. Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up.
2. Impaired sensation and circulatory disorders of the arms.
3. Impaired lymph drainage – including soft tissue swelling of uncertain origin, located remotely from the fitted aid.

Application instructions

Putting on the EpiTrain

To make the support easy to put on, first undo the strap and loosely attach it to the velour (1 / If using the strap). Fold the top part of the support outward so that the cut outs of the internal massage pads become visible (2). Then take hold of the support near the pad pocket and pull it over the elbow. Make sure that the outer part of the elbow (epicondyle) is positioned in the cut out of the pad, just the same as the inner part of the elbow (3). Then fold the upper edge of the support back into the original position. To increase the pain-relieving effect, clench your hand into a fist, tighten the strap moderately, and refasten the strap without any tension. After this, unclench your fist (4). If using the strap). As soon as the muscles are tense, a comfortable, relieving compression should be exerted where the muscle is attached. The strap can also be tightened as required, ensuring that it does not dig into the arm. The strap should be loosened or removed immediately in the event of tingling, blood congestion, swelling, or numbness.

Taking off the EpiTrain

To make the support easy to take off, undo the strap and loosely attach it to the velour. Then take hold of the support at the bottom edge near the pad pocket and pull it toward your hand. Please note: Only attach the strap to the velour surface provided and not to the knitted fabric as the surface of the knitted fabric may be damaged.

Product care instructions

The sewn-in label of the EpiTrain, which contains information on the product name, size, manufacturer, washing instructions and CE mark, is located on the inside of the knitted fabric.

All Train supports³ can be machine washed separately at 30 °C with liquid detergent. Please note the following when washing your Train products:

1. For products with Velcro® straps: Remove the straps if possible, or fasten them securely at the place provided for this purpose.
2. If possible, also remove pads / inserts and their terrycloth covers before washing.

We recommend using a mesh bag. Air dry your Train support. Do not dry it in a dryer because this can compromise the knitted fabric.

Please wash your Train support regularly so that the compression of the knitted fabric can be retained. This product has been examined by our own quality control system. However, should you have any complaint, please contact your retailer or our customer service department.

Notes on reuse

The product is intended to be fitted only once and to one patient.

Warranty

The legal provisions of the country in which the product was purchased apply. Country-specific warranty regulations that govern the relationship between the retailer and the buyer also apply where relevant. If you believe that a claim may be made under the warranty, please first contact the retailer from which you purchased the product directly.

Please do not make any changes to the product yourself. This particularly applies to individual adjustments made by specialists. Please follow our instructions for use and care. These are based on our experience gained over many years and ensure

the functionality of our medically effective products for a long time. Only aids that function optimally can help you. Failure to follow the instructions may also restrict the warranty.

Disposal

- The product can be disposed of without any restrictions.
- For information regarding the options available for returning the packaging to ensure it is correctly recycled, please observe the instructions from the sales outlet where you purchased this product.

Version: 2017-08

¹ A specialist is any person who is authorized according to the state regulations for fitting and instruction in the use of supports and orthoses which are relevant to you.

² Orthosis = orthopaedic appliance used to stabilize, relieve, immobilize, control or correct the limbs or the torso

³ Standardized washing tests show that after 50 washing machine cycles full functionality of the Train active support is retained.

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