AUSTRIA

Bauerfeind Ges.m.b.H.
Hainburger Straße 33
1030 Wien
P +43 (0) 800 4430-130
F +43 (0) 800 4430-131
E info@bauerfeind.at

BENELUX

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en english

These instructions for use can also be downloaded from: www.bauerfeind.com/downloads

Dear customer,

thank you for choosing a product from Bauerfeind. GenuTrain is a high-quality medical product. Please read through these **instructions for use** carefully and if you have any questions contact your doctor, medical retailer or our Technical Service department.

Intended use

GenuTrain, the active knee support, produces its effect during movement. Here the knitted fabric and the pad compress and massage the soft tissues. The resulting enhanced circulation in the joint capsule and promotion of lymph drainage lead to reduced swelling, pain relief and improved function while, at the same time, taking the strain off the kneecap. The three-dimensional knit ensures an anatomically perfect fit for the GenuTrain. The compression-reduced support edges stop the support cutting into the leg.

Indications

- Irritation after an injury, operation or overexertion of tendons, muscles or ligaments
- Degenerative changes in the knee (osteoarthritis)
- Joint inflammation (arthritis)
- Feeling of instability

Risks of using this product

⚠ Important information

- GenuTrain is a prescribable product. In case you have received it by prescription from your doctor

or physiotherapist, the use and duration of use must be agreed with her / him before utilizing the product. In any case, instruction is required by the specialist¹ who provided you with the product. GenuTrain should only be applied in accordance with the instructions contained in these usage directions and for the listed areas of application.

- No liability is accepted for improper use of the product.
- Changes must not be made for the product. Non compliance can reduce the performance of the product and thus no liability will be accepted.
- Should you notice any unusual changes (e.g. increased symptoms), please contact your medical advisor without delay.
- A combination with other products (e.g. compression stockings) must be discussed beforehand with your medical advisor.
- The product's unique action is produced primarily during physical activity. The product should be removed during prolonged rest breaks (e.g. sleeping, continuous sitting).
- Avoid the use of ointments, lotions or any substances including grease or acids.
- Side effects, that affect the entire organism, have not been reported to date. Correct fitting is assumed. Any supports and orthoses² applied externally to the body can, if tightened excessively, lead to local pressure signs or, rarely, constrict the underlying blood vessels or nerves.

⚠ Contains natural rubber latex (Elastodiene) which can cause skin irritation in persons with an allergy. Take off the support if you notice any allergic reaction.

Contra-indications

Hypersensitive reactions harmful to health have not been reported to date. In the following conditions such aids should only be fitted and applied after consultation with your medical advisor:

- Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up.
- Varicose veins.
- Impaired sensation and circulatory disorders of the legs (e.g. diabetes mellitus).
- Impaired lymph drainage – including soft tissue swellings of uncertain origin located remotely from the fitted aid.

Application instructions

Fitting GenuTrain (Illustrated)

- Firmly grip the top of the support with both hands (one either side of the lateral spines) at the level of the ring-shaped donning aid and pull over the knee joint until the knee cap is free from pressure and is surrounded by the silicone insert. **1 2 3**
- Adjust as required until a snug fit is achieved. **4**
- In contact sports a protective covering should be applied.

Removing GenuTrain

To remove the support, pull down at the bottom of the support on both sides near the stays. To avoid damaging the support, do not turn it inside out! **5**

Cleaning instructions

The sewn-in label of the GenuTrain, which contains information on the product name, size, manufacturer, washing instructions and CE mark, is located on the upper edge of the support.

⚠ All Train products³ can be machine washed separately at 30 °C with liquid detergent. Please note the following when washing your Train products:

- For products with Velcro® straps: Remove the straps if possible, or fasten them securely at the place provided for this purpose.
- If possible, also remove pads / inserts and their terrycloth covers before washing.

We recommend using a mesh bag (spin cycle: maximum of 500 revolutions per minute). Air dry your Train product. Do not dry it in a dryer because this can compromise the knitted fabric.

Please wash your Train product regularly so that the compression of the knitted fabric can be retained. This product has been tested by our in-house quality management system. If you have any complaints, please contact your medical retailer only. Please note that only clean goods can be dealt with.

Notes on reuse

The product is intended to be fitted only once and to one patient.

Warranty

The legal provisions of the country in which the product was purchased apply. Country-specific warranty regulations that govern the relationship between the retailer and the buyer also apply where relevant. If you believe that a claim may be made under the warranty, please first contact the retailer from which you purchased the product directly.

Please do not make any changes to the product yourself. This particularly applies to individual adjustments made by specialists. Please follow our instructions for use and care. These are based on our experience gained over many years and ensure the functionality of our medically effective products for a long time. Only aids that function optimally can help you. Failure to follow the instructions may also restrict the warranty.

Disposal

The product can be disposed of without any restrictions.

Version: 2017-07

¹ A specialist is any person who is authorized according to the state regulations for fitting and instruction in the use of supports and orthoses which are relevant to you.

² Orthosis = orthopaedic appliance used to stabilize, relieve, immobilize, control or correct the limbs or the torso

³ Standardized washing tests show that after 50 washing machine cycles full functionality of the Train active support is retained. (Liquid detergent was used.)

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