

# HeelZup® Cloud

## Heel Suspension and Leg Elevation Cushion

One of the leading challenges for the wound care community is heel pressure ulceration. Heels are naturally susceptible to pressure ulcers and even the most advanced support surfaces prove to be ineffective at prevention and treatment. Existing modalities of heel pressure ulcer prevention and treatment have many drawbacks. Heel “boots” require frequent removal to inspect the skin integrity of the lower leg. Pillows, which slide out of place and compress very easily are also a high maintenance and unreliable solution to the problem. The HeelZup® Cloud Heel Suspension Cushion provides the most practical solution to this complex problem:

- Complies with NPUAP and AHRQ Guidelines for heel pressure offloading.
- Suspends the heels a minimum of one inch off of the surface, while supporting the entire elevated area of the legs on a plush bed of poly-fiber and viscoelastic foam.
- Provides a high degree of comfort in both the supine and 30-degree side lying positions.
- Stays neatly in place and does not require constant maintenance and adjustment.
- Recovery Knit cover reduces friction and shear, while the No-Slip bottom material helps the cushion to stay neatly in place. No need for constant maintenance and adjustment.
- Manufactured of high-density, anti-microbial, hypo-allergenic and latex-free materials.
- Available in both single patient use and reusable models.
- Staff education is continually reinforced by illustrated instructions on the cover.



Phone: 866.736.8484  
Fax: 941.359.6939  
www.heelzup.com

**Intensive Therapeutics, Inc.**

P.O. Box 488  
Sarasota, FL  
34230

## Specifications:



### **HeelZup® Cloud Solo - For Single Patient and Home Use:**

Manufactured of high-density, anti-microbial, hypoallergenic VE foam and thermobonded polyester fiber. The standard width is 30". The Solo version is covered with a lightweight coated nylon fabric with a Velcro button enclosure. The cover can be wiped down or removed and laundered in cold water on a gentle cycle. HeelZup® Cloud Solo meets or exceeds the specifications in California Upholstery Provisions as stated in Technical Bulletin #117.



### **HeelZup® Cloud Reusable:**

Manufactured of high-density, anti-microbial, hypoallergenic VE foam and thermobonded polyester fiber. The standard width is 30". A moisture proof, reusable coated four-way stretch fabric cover with No-Slip bottom is sewn onto the cushion. A waterfall flapped zipper enclosure prevents moisture ingress through the zipper. The cover can be cleaned with standard cleaning procedures, using properly diluted solutions. HeelZup® Cloud meets or exceeds specifications in California Upholstery Provisions as stated in Technical Bulletin #117.

## Ordering Information:

Order Number	Description	Height Range	Weight Limit
Cloud-SR	HeelZup®Cloud Reusable, Small	5'4" and Under	300 Pounds
Cloud-MR	HeelZup®Cloud Reusable, Medium	5'5" - 5'10"	300 Pounds
Cloud-LR	HeelZup®Cloud Reusable, Large	5'11" and Up	300 Pounds
Cloud-SD	HeelZup®Cloud Solo, Small	5'4" and Under	300 Pounds
Cloud-MD	HeelZup®Cloud Solo, Medium	5'5" - 5'10"	300 Pounds
Cloud-LD	HeelZup®Cloud Solo, Large	5'11" and Up	300 Pounds



Comfortable and effective in both the supine and 30-degree side lying positions.



Phone: 866.736.8484  
 Fax: 941.359.6939  
[www.heelzup.com](http://www.heelzup.com)

## Warranty:

This warranty applies only to defects in structure, workmanship and materials and only to damages arising from normal hospital usage on a non-prorated basis for a period of six (6) months for solo models and three (3) years for reusable models from the date of purchase. Damages arising from abnormal use such as those caused by needle punctures, burns, chemicals, negligent use or improper care or cleaning are excluded from coverage. Manufacturer will pay shipping charges incurred in connection with this warranty during the first year, thereafter, those charges are the facilities' or dealer's sole responsibility.