


Homecraft Raised Toilet Seat with Handles

0817-16-331

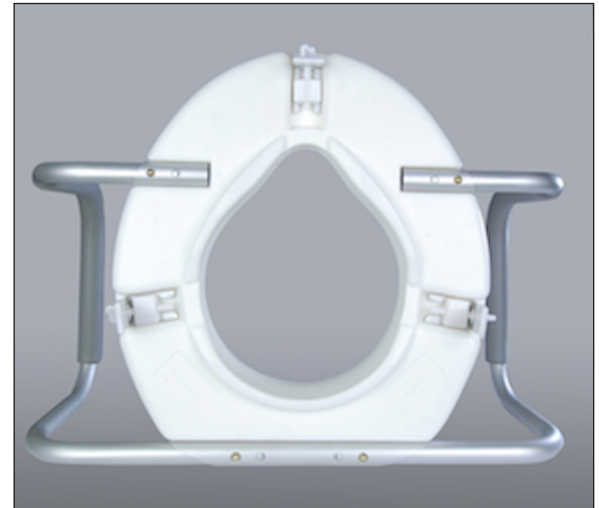

BEFORE INSTALLATION

Remove the Raised Toilet Seat with Handles from carton and read all instructions carefully before attempting to install product. Be sure arms are locked into place, by pulling them.

INTENDED USES

Designed to allow for, or increase independence when using the toilet, particularly by compensating for impaired movement.

INSTALLATION INSTRUCTIONS

- 1) Raise the toilet seat that is attached to the toilet. The Raised Toilet Seat with Handles should be put onto the toilet bowl.
- 2) Turn the knobs until the brackets are farther apart than the width of the toilet bowl.
- 3) Turn the Raised Toilet Seat with Handles over and place it on the toilet bowl with the narrow cutout at the front and the brackets near the back. Position the seat with the front lip on the seat against the outside front of the bowl.

4) Tighten both brackets until they are snug against the bowl.

5) To ensure that the Raised Toilet Seat with Handles is secure, regular checks of the brackets should be made. Spare brackets are available if required.

PRECAUTIONS

The Raised Toilet Seat with Handles is not to be used to support more than 300 lbs.

Be sure the Raised Toilet Seat with Handles is securely in place before putting weight on it.

Before each use, make sure there is full contact between the rubber pad on the bracket and the toilet bowl.

Under no circumstances should you stand on the Raised Toilet Seat with Handles.

CLEANING

The Raised Toilet Seat with Handles can be cleaned with soap and hot water (up to 85°C). Adjusting screws can be popped out of the brackets for cleaning.

Limited Product Warranty

Performance Health Supply, Inc. warrants that all products shall be free from manufacturing defects for a period of one year from the date of shipment. OTHER THAN THE FOREGOING WARRANTY, Performance Health Supply, Inc. MAKES NO REPRESENTATION OR WARRANTY OF ANY KIND, EXPRESS OR IMPLIED, AS TO MERCHANTABILITY, FITNESS FOR A PARTICULAR PURPOSE OR ANY OTHER MATTER RELATING TO THE PRODUCT. Customer must notify Performance Health Supply, Inc. of any manufacturing defect within five days of discovery of the defect. Upon a determination by Performance Health Supply, Inc. that the product is defective, Performance Health Supply, Inc. sole and exclusive obligation shall be to provide customer with a replacement product or a full refund of the purchase price, at the sole discretion of Performance Health Supply, Inc., and to reimburse customer for any return freight costs. In no event shall Performance Health Supply, Inc. be liable for any other costs or expenses including lost profits or direct, indirect, consequential, incidental or punitive damages. The warranty set forth herein shall govern all purchases, notwithstanding any inconsistent terms and conditions of customer's purchase order.

**PERFORMANCE
HEALTH**

Distributed by:

Performance Health Supply, Inc.
28100 Torch Parkway,
Ste. 700
Warrenville, IL 60555, USA
Phone: 800-323-5547
Direct Phone: 630-393-6000
Fax: 800-547-4333
Direct Fax: 855-393-7600

Manufactured for:

Performance Health Supply, Inc.
W68 N158 Evergreen Blvd.
Cedarburg, WI 53012, USA
Phone: 800-323-5547
Fax: 800-547-4333

Authorized European Representative

Performance Health
International, LTD.
Nunn Brook Road
Huthwaite, Sutton-in-Ashfield
Nottinghamshire, NG17 2HU, UK

United Kingdom

Phone: 03444 124 330
Fax: 03448 730 100
International
Phone: +44 (0) 1623 448 706
Fax: +44 (0) 1623 448 784

Rev. December 2016
© Performance Health Supply, Inc.
081716331