Rolyan® MFC® Unilateral Shoulder Orthosis:

A545-21, 22, 31, 32, 41, and 42

Rolyan® MFC® II Unilateral Shoulder Orthosis:

A545-51, 52, 61, 62, 71, and 72


A

B

C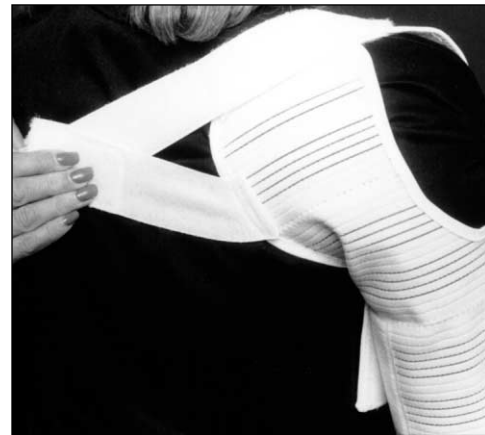

D

Be sure to note which product you are using and follow the instructions that apply to that product. Unless otherwise stated, these instructions apply to both of the products.

INDICATIONS

- To support weak shoulder muscles and strained tendons and ligaments
- To position scapula in retraction and limit shoulder rotation and abduction

CONTRAINDICATIONS

- Not in the presence of acute fracture

PRECAUTIONS

- **These products are to be fitted initially by a healthcare professional who is familiar with the purpose for which they are prescribed. The healthcare professional is responsible for providing wearing instructions and precautions to other healthcare**

practitioners, care providers involved in the patient's care, and the patient.

- **If unusual swelling, skin discoloration or discomfort occurs, use should be discontinued and a healthcare professional consulted.**
- **Be careful that the orthosis and straps are not so tight that they interfere with circulation.**

Rolyan MFC Unilateral Shoulder Orthosis

- **This product contains natural rubber latex, which may cause allergic reactions.**

CARE AND CLEANING

Fasten hook-and-loop closures. Hand wash with lukewarm water and mild soap. Wrap in towel to remove excess moisture and air dry. Be sure orthosis is thoroughly dry before reapplication.

APPLICATION

1. With all fasteners open, place arm cuff over affected shoulder so the D-ring lies on the chest (A).
2. Wrap arm cuff snugly around arm and fasten hook-and-loop closure. Position cuff as far distal as possible without inhibiting flexion and extension of elbow.
3. Pull the anterior shoulder strap over the shoulder and fasten to the posterior flap on the arm cuff (B).
4. Pull long strap across back, under unaffected arm, and across chest. Thread strap through D-ring and secure hook-and-loop fastener (C). Do not secure strap to axillary pad.
5. Secure short scapular control strap to long back strap (D).

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