

Homecraft Deluxe Bariatric Drop Arm Commode

0817-06-381



INSTRUCTIONS

1. Before use, adjust height of each leg to your specific needs, making sure all legs are at the same height. When desired height is reached, lock legs into place by aligning buttons into proper holes.
2. Be sure legs are locked into place by pushing down on, or by pulling legs.
3. Slide pail in from front of commode.

PRECAUTIONS

1. Do NOT install commode without first reading and understanding this assembly and instruction sheet. If you cannot understand these instructions, contact a healthcare, dealer or technical professional before attempting to install. Otherwise, injury or damage may occur.
2. Inspect all parts for shipping damage before assembly. If there is shipping damage—DO NOT USE. Contact dealer/carrier for further instructions.

3. Make certain that the height adjusting snap buttons fully protrude through the same respective hole of each leg extension. This will ensure that the leg extensions are securely locked in position and an even height is achieved.
4. Ensure ALL screws, nuts and/or bolts are tightened.
5. Check the rubber tips on the leg extensions for rips, wear, or to see if they are missing. Immediately replace any or all if any of these imperfections exist.
6. All four leg extensions with rubber tips MUST touch the floor simultaneously at all times.
7. Users with limited physical strength should be supervised or assisted while using commode.
8. Do not exceed user weight capacity of 1,000 lbs.

Limited Product Warranty

Performance Health Supply, Inc. warrants that all products shall be free from manufacturing defects for a period of one year from the date of shipment. OTHER THAN THE FOREGOING WARRANTY, PERFORMANCE HEALTH SUPPLY, INC. MAKES NO REPRESENTATION OR WARRANTY OF ANY KIND, EXPRESS OR IMPLIED, AS TO MERCHANTABILITY, FITNESS FOR A PARTICULAR PURPOSE OR ANY OTHER MATTER RELATING TO THE PRODUCT. Customer must notify Performance Health Supply, Inc. of any manufacturing defect within five days of discovery of the defect. Upon a determination by Performance Health Supply, Inc. that the product is defective, Performance Health Supply, Inc. sole and exclusive obligation shall be to provide customer with a replacement product or a full refund of the purchase price, at the sole discretion of Performance Health Supply, Inc., and to reimburse customer for any return freight costs. In no event shall Performance Health Supply, Inc. be liable for any other costs or expenses including lost profits or direct, indirect, consequential, incidental or punitive damages. The warranty set forth herein shall govern all purchases, notwithstanding any inconsistent terms and conditions of customer's purchase order.



Distributed by:
Performance Health Supply, Inc.
28100 Torch Parkway,
Ste. 700
Warrenville, IL 60555, USA
Phone: 800-323-5547
Direct Phone: 630-393-6000
Fax: 800-547-4333
Direct Fax: 855-393-7600

Manufactured for:
Performance Health Supply, Inc.
W68 N158 Evergreen Blvd.
Cedarburg, WI 53012, USA
Phone: 800-323-5547
Fax: 800-547-4333

Authorized European Representative
Performance Health International, LTD.
Nunn Brook Road
Huthwaite, Sutton-in-Ashfield
Nottinghamshire, NG17 2HU, UK

United Kingdom
Phone: 03444 124 330
Fax: 03448 730 100
International
Phone: +44 (0) 1623 448 706
Fax: +44 (0) 1623 448 784

Rev. December 2016
© Performance Health Supply, Inc.
081706589