



The  
personal fit

## 6 EASY STEPS TO ATTACH VELCRO TO ORFICAST NO SEWING, GLUING OR IRONING! ONLY DRY HEAT.

1. Prepare two strips of Orficast 1" x 2", a small piece of adhesive backed hook and a longer piece of loop strapping.
2. Using dry heat, heat up one strip of Orficast and the adhesive side of the hook and press together. The Orficast acts as an interface.
3. Trim the excess Orficast and adhere to the orthosis using dry heat on both the orthosis and the prepared hook.
4. Heat the second strip of Orficast and wrap around one end of the loop strapping. Trim corners.
5. Heat the orthosis and the prepared end of the loop strap with dry heat and press together.
6. Alternatively, heat a thin strip of Orficast and use as binding tape around the edges of the hook and the loop after these have been bonded to the orthosis.



WATCH VIDEO



[www.orfit.com](http://www.orfit.com)



**ORFIT INDUSTRIES**

Vosveld 9A | B-2110 Wijnegem | Belgium  
T (+32) (0)3 326 20 26  
[welcome@orfit.com](mailto:welcome@orfit.com)

**ORFIT INDUSTRIES AMERICA**

350 Jericho Turnpike, Suite 302 | Jericho, New York 11753  
T 516-935-8500 | F 516-935-8505  
[sales-us@orfit.com](mailto:sales-us@orfit.com)