IOMEDFirst in Iontophoresis "Quality . . .
Measured in Outcomes"**ENGLISH****Controlled Power Iontophoresis System • The Mobile Solution**
Directions for use with Companion 80®**A. INDICATIONS**

- Companion 80® is indicated for the administration of soluble salts or other drugs into the body for medical purposes as an alternative to hypodermic injection.

B. CONTRAINDICATIONS

- Companion 80 is contraindicated for use on patients with the following:
 - Cardiac pacemakers or other sensitive implanted devices
 - Known sensitivity to drug to be administered
- It is contraindicated for use over damaged or denuded skin or other recent scar tissue.
- It is also contraindicated for treatment across either the temporal region or the orbital region of the patient.

C. DESCRIPTION

Companion 80 is a disposable, single-use, iontophoresis system with an internal battery and current limiting circuitry. In use, the Companion 80 provides constant drug delivery for an 80 mA-minute treatment and can deliver both negatively and positively charged water-soluble drugs. The Companion 80 operates at a low current and is worn for 24 hours to deliver the desired dose. The Companion 80 is designed to begin a treatment once it is hydrated and applied to the skin and stop the treatment at approximately 80 mA-minutes.

D. PRECAUTIONS

- Care must be taken when operating this equipment adjacent to or stacked with other equipment. Potential electromagnetic or other interference could occur to this or to the other equipment. Care should be taken to minimize this interference by not using other equipment in conjunction with it.
- Do not wear electrode or controller during Magnetic Resonance Imaging (MRI) scans as this may result in metal overheating and causing skin burns in the area of the patch.
- Potential systemic adverse effects may result from use of this device. Drugs or solutions delivered with this device have the potential to reach the blood stream and cause systemic effects. Carefully read all labeling of the drug or solution used with this device to understand all potential adverse effects and to ensure appropriate dosing information. If systemic manifestation occur, refer to the drug or solution labeling for appropriate action.

E. CAUTION

U.S. Federal Law restricts this device to sale by or on the order of a physician.

**Single-use only**

- Store at ambient temperature 15°-30°C (59°-86°F)
- DO NOT expose to temperatures above 50°C (122°F)

Approximate Dosage: 80 mA-minutes

Recommended Wear Time: 24 hours

F. CARTON CONTENTS

- 6 Companion 80 Iontophoresis Systems
- 6 Saline (NaCl) Ampules
- 6 Alcohol Preps

Product	Approx. Fill Volume	Active Area	Order Number
Companion 80	1.1ml	6.45cm ²	5000044

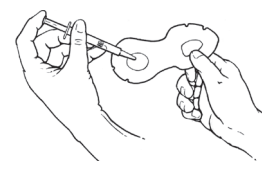
G. PREPARING THE PATIENT

- Advise the patient that iontophoresis causes mild tingling, pricking and/or a warm sensation. This is normal and should be anticipated by the patient.
- Advise patient that iontophoresis has the potential to result in skin irritation and/or burns.
 - CAUTION** – Direct current may result in transient erythema under either pad. It generally resolves within a few hours to a few days.
 - CAUTION** – Use caution when treating patients with sensitive skin or who may have difficulty healing.
- Advise patient to remove and dispose of the Companion 80 if they feel any discomfort during treatment.
- Advise patient to remove any jewelry that may come in contact with the Companion 80.
 - NOTE** – Jewelry in contact with either the Companion 80 during treatment may result in burns.

DIRECTIONS FOR USE**H. PREPARING THE COMPANION 80**

- Remove the following components from the carton:
 - Companion 80 Treatment Kit
 - Saline Ampule
 - Alcohol Prep
- Tear open sealed treatment kit and remove Companion 80.

- Hydrate pads with approximately 1.1ml of fluid, eliminating any dry spots (Step 1).

**Step 1**

- Use negatively charged medications on the negative (-) pad and positively charged medications on the positive (+) pad. Use saline solution provided on the opposite pad.

NOTE – To ensure proper pad hydration and performance:

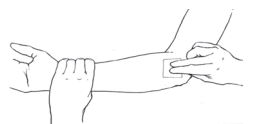
- DO NOT fill pads on patient
 - DO NOT over- or under-fill pads
 - DO NOT use drugs that are not water-soluble
 - DO NOT use components that appear altered or damaged
- NOTE** – For information on "needle free" filling techniques, contact your authorized DJO representative or DJO Customer Service.

- Make sure that the treatment site has intact skin.
 - NOTE** – DO NOT apply to damaged or denuded skin or other recent scar tissue; skin with ingrown hair, pimples, razor nicks or skin with wounds that have not healed.

WARNING – Failure to follow these guidelines can cause skin irritation or burns.

CAUTION – DO NOT treat around the orbital region of the patient.

- Clean the treatment site thoroughly with alcohol prep by rubbing for six to eight seconds to remove dry skin, oils and other contaminants (Step 2).

**Step 2**

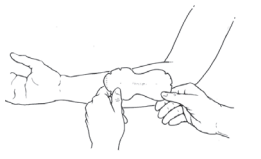
CAUTION – Failure to clean skin thoroughly can cause excessive skin irritation or burns.

NOTE – Clip hair if necessary to improve skin contact. DO NOT shave. Dirt accumulation can cause poor contact

- Allow site to dry completely.
- Dirt accumulation can cause poor contact

I. ADMINISTERING TREATMENT

- Remove the backing from the adhesive and apply the drug pad on the selected treatment site and secure it by pressing on the adhesive border (Step 3).
 - NOTE** – Avoid pressing directly on the hydrated pads; excessive pressure can cause drug to leak.
 - CAUTION** – DO NOT tape, bind or compress the Companion 80 during treatment.
- Advise patient to wear the Companion 80 as directed.
 - Note** – DO NOT immerse or expose Companion 80 to direct water stream.
- At the completion of treatment, advise the patient to remove and discard the Companion 80.
 - CAUTION** – DO NOT REUSE patches that have been previously used as these patches have been designed for single use only. Reuse will not be effective.
 - NOTE** – Keep out of reach of children.

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