en Instructions for use for Tribute® Wrap Wrist to Axilla Adjustable foam compression wrap

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



Symbol glossary provided at: www.Lohmann-Rauscher.com/symbolsglossary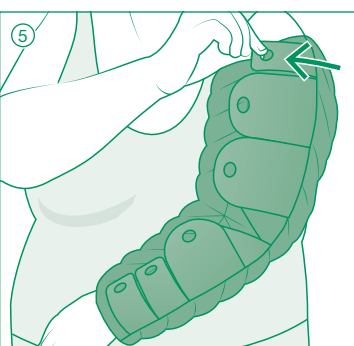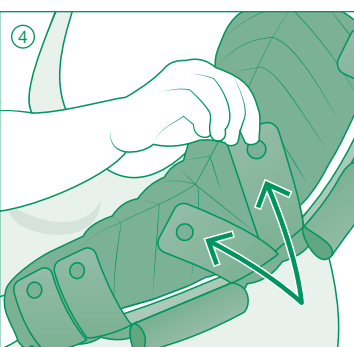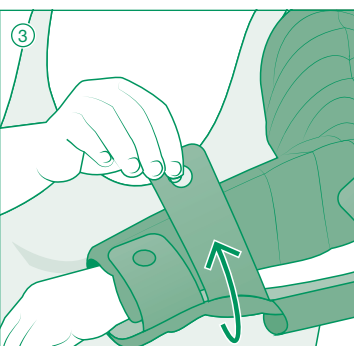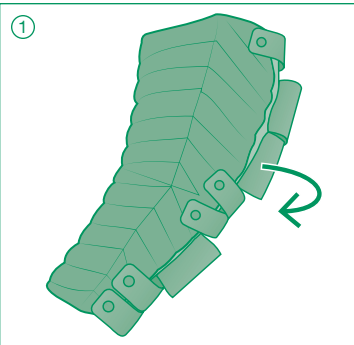

en | Instructions for use

Thank you for choosing Tribute® Wrap!

Tribute® Wrap Wrist to Axilla is an off-the-shelf adjustable foam garment for the self-management of lymphedema or edema of the arm during the evening, night, or low activity. Tribute Wrap comfortably adjusts to your unique shape to help maintain gains made during therapy, while allowing the flexibility to support further reduction.

Please consult a trained medical professional prior to first use.

Indications and patient groups

- Various stages of lymphedema
- Edema

Intended uses

- Helps maintain reduced volume and shape by supporting extremities
- Management of various stages of lymphedema
- Management of edema
- Assists in softening of indurated/fibrotic tissue

Contraindications

- Arterial insufficiency or ABPI < 0.8
- Moderate to severe peripheral arterial disease
- Acute DVT
- Untreated congestive heart failure
- Untreated cancer
- Untreated infection (e.g. cellulitis)
- Known allergy and/or hypersensitivity to any of the product components
- Severe cognitive impairment

Precautions

- Impaired sensation (peripheral neuropathy)
- Cognitive impairment
- Any open wounds should be dressed prior to application

Side effects

In (very) rare cases, skin irritations and/or allergies may occur.

Serious incident

In case of serious incident, contact the manufacturer and the responsible health authorities.

Donning instructions

- Fasten the wrist, upper arm, and angled elbow straps at the edge of the garment. Roll the remaining straps back loosely.
- Slide your arm through the wide end of the garment ensuring your elbow is centered between the two angled straps.
- Pull the narrow strap(s) at your wrist using the finger grip and fasten to the garment. Adjust for fit and comfort.
- Pull the two narrow straps at your elbow using the finger grip and fasten to the garment at an angle. Adjust for fit and comfort.
- Fasten the remaining straps, starting closest to the wrist, using the same process.

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Lohmann & Rauscher B.V
Lohmann & Rauscher Limited