




7102990 Days single Button Folding Walker Adult No Wheels  
7102991 Days single Button Folding Walker Adult 5"Wheels

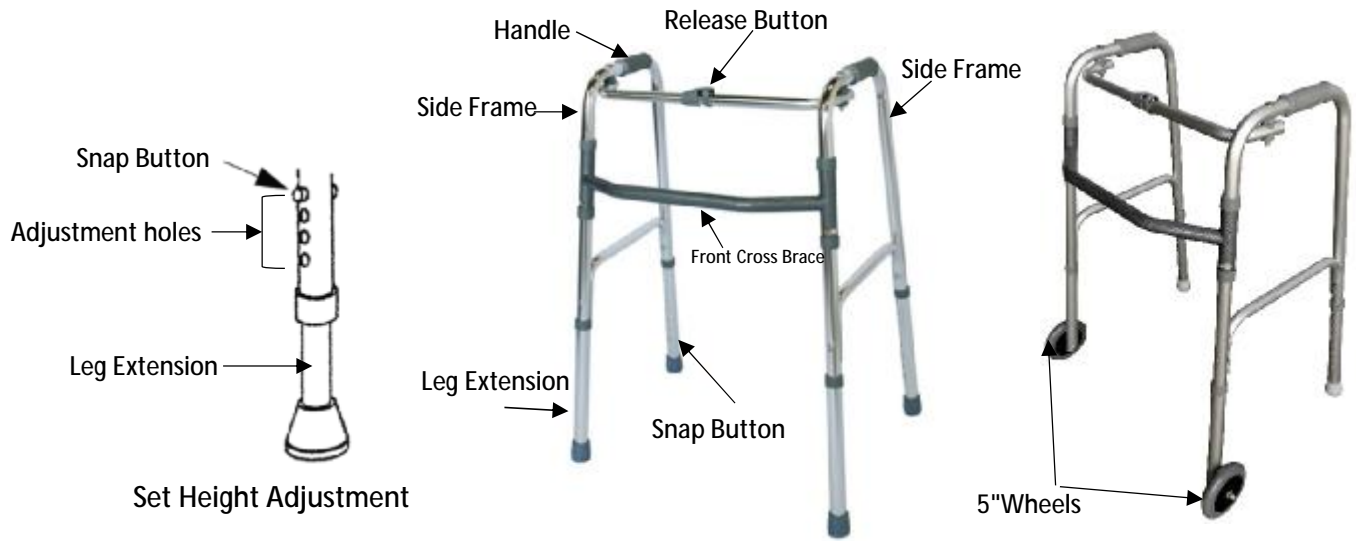

**Pre-Assembly**

Inspect the product for damage that may have occurred during shipping. In the case of defects, contact us immediately.

Read the following instructions before attempting usage. If the directions are not clear, contact a healthcare professional or technician.

To achieve the proper adjustments and guidelines for use, consult with a therapist or other medical professional.

**Opening/Folding the Walker**

To open, push down the release button and swing the side frames open until the button pop back up. At the same time, you will hear a "click," indicating that the walker is open all the way. The open position will be securely locked until the release button is used again for folding. Press the buttons and fold in one side of frame and then the other.

**Adjust Leg Extensions**

Determine the desired overall height of the walker: The height of walker should be at the wrist when the user's arms down.

Select the adjustment hole on the leg of the frame that will achieve the desired height. Push the snap button on the leg of the frame, slide the leg extension on the chosen hole and release the button to lock in place. Repeat steps for each leg extension.

Confirm that the chosen adjustment holes are at the same height and that snap buttons are fully positioned through the holes.

**Precautions**

Do not exceed the maximum user weight of 300 Lbs/136kg. The walker stability and walking; do not use it to support full body weight.

Before each use: ensure snap buttons protrude all the way through the holes. All leg extensions MUST be set at the same height for safe operation.

Before using, double check that walker has been fully opened and is level to the floor or ground. The walker is not designed to hold items. Hanging objects on the walker could tip it over and potentially causing personal injury or damage.

Periodically inspect rubber tips for wear and tear. If tips are damaged, replace them immediately.

**Limited Product Warranty**

Performance Health warrants that all products shall be free from manufacturing defects for a period of one year from the date of shipment. OTHER THAN THE FOREGOING WARRANTY, PERFORMANCE HEALTH MAKES NO REPRESENTATION OR WARRANTY OF ANY KIND, EXPRESS OR IMPLIED, AS TO MERCHANTABILITY, FITNESS FOR A PARTICULAR PURPOSE OR ANY OTHER MATTER RELATING TO THE PRODUCT. Customer must notify Performance Health of any manufacturing defect within five days of discovery of the defect. Upon a determination by Performance Health that the product is defective, Performance Health sole and exclusive obligation shall be to provide customer with a replacement product or a full refund of the purchase price, at the sole discretion of Performance Health, and to reimburse customer for any return freight costs. In no event shall Performance Health be liable for any other costs or expenses including lost profits or direct, indirect, consequential, incidental or punitive damages. The warranty set forth herein shall govern all purchases, notwithstanding any inconsistent terms and conditions of customer's purchase order.

Serious events experienced from using this product should be reported immediately to Performance Health and to the local Competent Authority.

Symbols Glossary located at <http://www.performancehealth.com/medical-device-glossary>

Distributed in the US by:  
Performance Health Supply Inc.  
W68 N 158 Evergreen Blvd.  
Cedarburg, WI 53012 USA  
Phone: +1 800-323-5547  
[www.performancehealth.com](http://www.performancehealth.com)

Distributed in Canada by:  
Performance Health Canada, Inc.  
6675 Millcreek Drive, Unit 3  
Mississauga, ON L5N 5M4  
Phone: 800-665-9200  
[www.performancehealth.ca](http://www.performancehealth.ca)

Distributed in the UK by:  
Performance Health International Ltd.  
Nunn Brook Road, Huthwaite,  
Sutton-in-Ashfield, Nottinghamshire,  
NG17 2HU, UK.  
UK: Tel: 03448 730 035  
[www.performancehealth.co.uk](http://www.performancehealth.co.uk)  
International:  
Tel: +44 1623 448 706

JAN MAO INDUSTRIES CO., LTD  
Zhi Xin Road, Pingguo County Industrial Zone,  
Baise City, Guangxi, China (post code: 531400)

Y. Sung Handelsvertretung  
Duesselthaler Str. 24  
40211 Duesseldorf Germany



Effective: January 2021  
©Performance Health