

Rolyan® Boot

5997 Rolyan Walkable Universal Boot
6188-01 Rolyan Comfi-Boot™ Plus 1 Universal Boot
6187-01 Rolyan Comfi-Boot™ Plus 2 Universal Boot

6920-03 Rolyan Durable Universal Boot
6921-02 Rolyan Plus Universal Boot with Foam Liner
6922-02 Rolyan Plus Universal Boot with Fleece Liner



Indications

Used to off-load heel and to prevent foot drop and hip rotation.

Contraindications

Not to be used in the presence of fixed contracture of the knee or the hip.

Not to be used with excessive contractures per physician's instruction.

Not to be used over open wounds.

Precautions

This splint is to be fitted initially by a healthcare professional who is familiar with the purpose for which they are prescribed. The healthcare professional is responsible for providing wearing instructions and precautions to other healthcare practitioners and care providers involved in the patient's care and patient.

Rolyan Boot cannot be used without soft covers.

Rolyan Boot cannot be used for ambulation or while weight bearing without walker attachment.

Rolyan Boot is single-patient use only.

If unusual redness, swelling, chafing, bruising or skin irritation occurs, discontinue use and a healthcare professional should be consulted.

Be careful that the straps are not so tight that they interfere with circulation.

Fitting Instructions

Remove contents from the package, and open all closures on the front of the boot.

Toe post and toe post cover can be removed from boot if necessary.

Ideal fitting of Rolyan Boot is with patient in a supine position (on their back) with knee bent so the plantar side (bottom) of the foot is resting in an even, flat position.

Carefully lift the patient's leg, and put the open boot in the natural position under the calf and foot. Ensure the foot is fully down into the

splint. It needs to be snug against the liner of the splint, making contact with the liner.

Ensure that the heel is not in contact with any of the hard surfaces or receiving pressure. The heel must be in an unweighted, off-loaded position.

Ensure there are no large wrinkles in the liner on the top of the boot. Use the hook and loop closures on the top cover to secure the calf area; then move to the foot area.

Ensure that you have good contact between the liner and the bottom of the foot. Carefully maintain the relationship between the heel and the foot. Secure the cover by connecting the hook and loop materials together.

Place an index finger between the boot cover and the patient's skin to ensure that the liner/cover is not too snug. If it is too snug, unhook the hook and loop closure and readjust.

If the boot is too snug, there are two optional expander pads included. The larger one is used on calf portion and smaller is used on foot portion. Both attach with hook and loop to extend to the circumference that the boot can accommodate.

De-Rotation Bar

To be used to help prevent excessive external rotation.

Located on the posterior portion of the boot shell, held in place with a finger nut.

Unlock the bar by loosening the finger nut.

Rotate the bar until it points away from the patient. Lock the nut back into position.

Removing/Installing Walker Bottom Pad

(Does not apply to the Comfi-Boot Plus Boots.)

To remove: boot needs to be off of patient. Pull out toe piece. Pull foot pad away from boot bottom. Unfasten hook & loop around heel portion and remove walker pad.

To install: boot needs to be off of patient and toe piece removed. Fasten hook & loop straps

from walker bottom pad around heel portion of shell. Bring together loop area, bottom of the foot portion of the cover, onto hook area of walker bottom. The loop strap, found at distal end of cover by toes, needs to be placed at distal end of walker pad, so it is past slit in walker bottom pad. Slide toe piece into slit of walker bottom pad, making sure that the toe piece goes through loop strap at distal end of cover.

Washing Instructions

Remove the soft cover and walker bottom from plastic frame. Engage all hook and loop closures.

Cover can be hand washed or machine washed on gentle cycle (in the laundry bag provided) with mild soap and water. Air dry.

Frame and walker bottom can be wiped off with damp cloth.

Air dry.

Limited Product Warranty

Patterson Medical warrants that all products shall be free from manufacturing defects for a period of one year from the date of shipment. OTHER THAN THE FOREGOING WARRANTY, PATTERSON MEDICAL MAKES NO REPRESENTATION OR WARRANTY OF ANY KIND, EXPRESS OR IMPLIED, AS TO MERCHANTABILITY, FITNESS FOR A PARTICULAR PURPOSE OR ANY OTHER MATTER RELATING TO THE PRODUCT. Customer must notify Patterson Medical of any manufacturing defect within five days of discovery of the defect. Upon a determination by Patterson Medical that the product is defective, Patterson Medical sole and exclusive obligation shall be to provide customer with a replacement product or a full refund of the purchase price, at the sole discretion of Patterson Medical, and to reimburse customer for any return freight costs. In no event shall Patterson Medical be liable for any other costs or expenses including lost profits or direct, indirect, consequential, incidental or punitive damages. The warranty set forth herein shall govern all purchases, notwithstanding any inconsistent terms and conditions of customer's purchase order.



PATTERSON
MEDICAL

Distributed by:

Patterson Medical

28100 Torch Parkway, Ste. 700
Warrenville, IL 60555
Phone: 800-323-5547
Direct Phone: 630-393-6000
Fax: 800-547-4333
Direct Fax: 855-480-1946

Manufactured for:

Patterson Medical

W68 N158 Evergreen Blvd.
Cedarburg, WI 53012
Phone: 800-323-5547
Fax: 800-547-4333

Authorized European

Representative

Patterson Medical, Ltd.

Nunn Brook Road
Huthwaite, Sutton-in-Ashfield
Nottinghamshire, NG17 2HU, UK

United Kingdom

Phone: 08444 124 330

Fax: 08448 730 100

International

Phone: +44 (0) 1623 448 706

Fax: +44 (0) 1623 448 784

Effective 2/14

Patterson Medical

Products, Inc.

99104.012