



Homecraft Shower Chair with Back and Arms

Maximum User Weight: 400lbs/180kg

Model 7103081

The following manual provides descriptions relating to the installation of the Homecraft Shower Chair with Back and Arms.

INSTALLATION INSTRUCTIONS

NOTE: Before assembly, ensure that all parts and quantities match parts list below. Inspect all components for shipping damage. If any parts are determined to be missing or damaged, contact your dealer/retailer for replacements. ⚠️ **WARNING!** DO NOT use this equipment without first reading and understanding the user manual. If you are unable to understand these instructions, contact a healthcare professional, dealer, or technical person before installing. Otherwise, injury or damage may occur. Symbols Glossary located at <http://www.performancehealth.com/medical-device-glossary>.

PARTS LIST

- A. Seat - 1 each
- B. Leg - 4 each
- C. Arms - 2 each
- D. Back - 1 each

ASSEMBLY INSTRUCTIONS

1. Place seat upside down on a flat surface. Insert each leg into the corner recess of the seat and rotate counter-clockwise until tight. Repeat step #1 for all 4 legs.
2. Place chair upright and insert arms into holes in side of seat until arm is fully engaged.
3. Insert back into the slot, which is in the rear of the seat, and insert back until fully engaged.
4. To adjust legs for proper height or uneven surfaces, grab the shower chair leg with one hand while rotating the bottom of the leg (where the suction cup attaches to the leg) clockwise. The leg has an adjustment window that indicates the level with a red marker. Turn to desired height and repeat for all legs.



CAUTIONS

- Ensure all 4 legs are at the correct height. The leg has an adjustment window that indicates the level with a red marker. Turn to desired height and repeat for all four legs.
- Only use the Shower Chair for its intended use. They are not replacements for ladders and are not exercise equipment or toys.
- The Shower Chair is not intended for use during examinations or treatments.
- Use only spare parts and accessories released by Performance Health.
- Do not carry out structural changes which are not approved by the manufacturer.
- The Shower Chair must not be used in environments in which flammable gases or vapours (e.g. from anaesthetics) can be expected.

CLEANING

- Small stains and dust are best removed with moist wipes.
- Larger stains can be removed by non-scouring, mild cleaning agents.
- Do not use tools with sharp edges (knives, metal scrapers) or aggressive solvents for cleaning.

MAINTENANCE

The Shower Chair is maintenance-free, however, should be checked monthly for damage or looseness.

Carry out the following test:

- Visual check of all parts for deformation, damage, wear and tear and corrosion.

The chair must not be used further if faults/defects are detected during use.

DISPOSAL

The Shower Chair consists of plastic parts.

Disposal of this material must be carried out according to the regional environmental and disposal provisions.

⚠️ RESTRICTIONS OF USE!

- Patients weighing more than 400lbs must not use the Shower Chair with Back and Arms.

LIMITED PRODUCT WARRANTY

Performance Health Supply, Inc. warrants that all products shall be free from manufacturing defects for a period of one year from the date of shipment. OTHER THAN THE FOREGOING WARRANTY, PERFORMANCE HEALTH SUPPLY, INC. MAKES NO REPRESENTATION OR WARRANTY OF ANY KIND, EXPRESS OR IMPLIED, AS TO MERCHANTABILITY, FITNESS FOR A PARTICULAR PURPOSE OR ANY OTHER MATTER RELATING TO THE PRODUCT. Customer must notify Performance Health Supply, Inc. of any manufacturing defect within five days of discovery of the defect. Upon a determination by Performance Health Supply, Inc. that the product is defective, Performance Health Supply, Inc. sole and exclusive obligation shall be to provide customer with a replacement product or a full refund of the purchase price, at the sole discretion of Performance Health Supply, Inc. , and to reimburse customer for any return freight costs. In no event shall Performance Health Supply, Inc. be liable for any other costs or expenses including lost profits or direct, indirect, consequential, incidental or punitive damages. The warranty set forth herein shall govern all purchases, notwithstanding any inconsistent terms and conditions of customer's purchase order.

Distributed in the USA by: Performance Health Supply, Inc.

W68 N 158 Evergreen Blvd.
Cedarburg, WI 53012 USA
Phone: 800-323-5547
www.performancehealth.com

Com-Bridge International Co., Ltd

No.66, Yu Long Road,Dong Sheng Town,Zhong Shan City,Guang Dong Province,China

