



**rolyan**<sup>®</sup>  
for Hand Therapy.

## SPLINT BATH CARE CARD



Refer to the your user manual

### Cleaning Your Splint Bath



**Frequency** – Two or more times monthly. Strava recommends each facility determine its cleaning frequency based on usage, water grade, visual inspection, and sanitation protocols that apply to your facility and/or JACHO.



**1 EVACUATE WATER**  
See user manual procedure.



**2 TURN UNIT OFF-ADD WATER & CLEANING SOLUTION**  
Fill unit just above the water sensor with a 2:1 ratio of tap water to cleaning solution.



**Cleaning Solution =  
Vinegar -OR- Coffee Machine/  
Commercial Descaler**



**3 TURN UNIT BACK ON**  
Run for 10 minutes @ 135°F/57°C.



**4 EVACUATE WATER**  
See user manual for procedure.



**5 TURN UNIT OFF**  
Unplug and allow unit cool down before proceeding.



**6 CLEAN INSIDE WATER BASIN**  
Using a Scotchbrite® Scour Pad or cloth, wipe the inside tub with warm water. To remove scum, waterlines, mineral deposits, etc., add a mild detergent. Be sure to rinse thoroughly and wipe dry. **Do NOT use bleach or extra abrasive products such as steel wool.**



**6 CLEAN INLINE FILTER**  
Remove the thermocouple inline filter found inside the water basin beneath the grate to clean debris. See **“quick tips”** section for removal and cleaning. Return the cleaned inline filter to the thermocouple.



**8 REFILL WITH WATER**  
Fill unit to operating level with tap water. See **“quick tips”** section if you use distilled water.



**9 RUN FOR 5 MIN.**  
Connect power, turn unit on, and prime unit (see **“quick tips”** section) to ensure circulation.


 Repeat procedures if necessary

## Cleaning The Glass





Over time, water vapor may cause calcium to build upon the glass and in the hinges. This is perfectly normal and easily cleaned with vinegar on a soft cloth or a commercial glass cleaner.

## Replacing the UV-C Bulb

 **Annually** – Recommend changing the bulb annually even when it is still working.

 **Purchase** – Replacement bulbs can be purchased from a Strava dealer.

 **Replacement** – Bulb replacement is simple and can be done by anyone, however, we recommend a qualified service technician if your organization has one.

 Check out the bulb replacement video at: [www.stravasolutions.com](http://www.stravasolutions.com)

## QUICK TIPS

### DISTILLED WATER

Tap water is required for the unit to run correctly. Follow these proportions if you are in a hard-water environment and require distilled water.

1 Cup Tap Water



- PER -



1 Gal.  
Distilled

### CLEAN INLINE FILTER

*Note: there are two filter versions depending on the model date.*

Loosen the filter with supplied Allen key. Wiggle the filter off of the thermocouple. Run under water and scrub out debris from the screen with a hard bristled brush utensil.

Fit the cleaned filter back on the thermocouple by wiggling it into place (screw heads facing out). Tighten the filter, ensuring that it is properly aligned and close to the hole.



### PRIMING

Priming removes any trapped air allowing the pump to engage. Insert priming bulb into the priming port. Squeeze and hold until the bulb is fully inflated. Release from the port. Once the water bubbles, the unit is primed. Repeat if necessary.



### FOR TECHNICAL SERVICE

email: [technicalsupport@stravasolutions.com](mailto:technicalsupport@stravasolutions.com)