

PERFORMANCE HEALTH®

PILL CRUSHER / PULVERIZER

Instructions for use

- When grinding tablets that are not easy to crush, please do not tighten them too hard and then rotate them backward. Please tighten and back off the spin and then repeat the grinding operation.
- It is not recommended to grind more than one tablet at a time, please do so in small quantities.

Cleaning and Maintenance

For regular maintenance, the Pill Crusher/Pulverizer can be cleaned with damp cloth or water.

NOTE: Do not use corrosive detergent.



PERFORMANCE HEALTH®

Pill Splitter

Instructions for use

- Cutting: Place the product on a flat surface and open the lid. Then place the tablet in the "V" area and close the lid. Press and hold the lid down to ensure that the knife on the lid cuts the tablet open.
- There are two independent compartments inside the product that can store two types of tablets. The remaining tablets after taking the medication can also be stored in this container for future use.
- After continuous use, the blade may become dull and the product needs to be replaced.



⚠️ CAUTION: Cut hazard sharp blade inside lid. Contact may result in injury. Always keep blades away from fingers and body. Handle with care. Keep out of reach of children.

⚠️ CAUTION: KEEP HANDS AND FINGERS OUT OF CRUSHING AREA TO PREVENT ACCIDENTAL INJURY.