

CryoDose TA

Topical Anesthetic Spray

Safe. Quick. Easy.

CryoDose TA temporarily controls pain associated with injections, starting IVs, venipuncture, minor surgical procedures, and minor athletic injuries.



Over 10,000 healthcare facilities have chosen CryoDose TA

CryoDose TA is a safe, cost-effective topical anesthetic spray that offers medical professionals a convenient solution for pain management. CryoDose TA's instant onset of action ensures that patients experience immediate pain relief helping the medical practitioner provide the highest level of care for the patient, improving overall satisfaction, and promoting better healthcare outcomes.

- FDA cleared, non-drug, instant topical anesthetic skin refrigerant
- Ingredients: pharmaceutical grade Pentafluoropropane (HFC-245fa) and Tetrafluoroethane (HFC-134a)
- Extensive indications for use, including intact mucous membranes and minor open wounds
- Intuitive nozzle provides accurate spray stream
- 100% evaporates and safely cools skin to well below freezing (-10°C to -12°C)
- Lots are tested to USP <61> and USP <62> standards by an outside, independent lab
- Proven safe for use after antiseptic cleaning in an aseptic procedure
- HAZMAT-free, non-flammable, and non-toxic; safe for use in confined spaces like exam rooms
- Can be used by a licensed healthcare provider, no doctor's order required
- Cost effective
- 100% satisfaction guarantee

CryoDose TA- Mist Spray

Item #7027118

CryoDose TA- Medium Stream

Item #7027119

*Now sprays 360°



Cost
Effective

Safe

Extensive
Indications
for Use

Do More with CryDose TA

CryoDose TA is intended for topical application to the skin, intact mucous membranes (oral cavity, nasal passageways, and lips), and minor open wounds. This pain management solution is both safe and highly effective, providing remarkable versatility for an extensive range of uses.

Minor surgical procedures

- I&D of Abscess, Paronychia, Bursa
- Lancing Boils
- Suturing, Stapling
- Removal of Staples, Sutures, Pins, and Other Hardware
- Wound Debridement
- Foreign Body Removal (e.g., Splinter)

Injections

- IV Starts
- Joint Injections
- PRP Injections
- Vaccinations
- Cosmetic Procedures— Fillers, Pre-lidocaine Injections, BOTOX®
- Venipuncture

Temporary relief of minor sports injuries

- Sprains
- Bruising
- Cuts and Abrasions

How to Use


Follow facility's protocol for cleaning the site with an antiseptic, have all equipment ready for the procedure, and then apply spray prior to performing the procedure.

The anesthetic numbing effect can last up to 60 seconds. If a procedure takes longer than anticipated, reapply as necessary.




1 Hold can 3"- 7" from procedure site.

Refer to application instructions for full details.

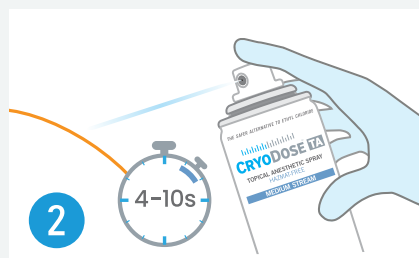

2 Spray steady 4 to 10 seconds, stop if skin turns white.

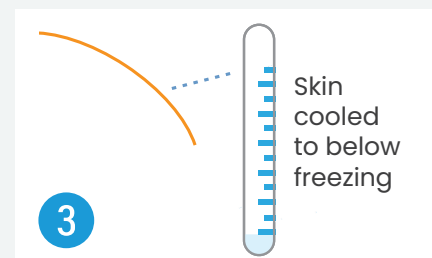

3 After spraying site, immediately perform procedure. Anesthetic effect lasts about 1 minute.