

Sky Medical Ankle Foot Orthosis – Posterior Leaf Spring (SM-AFO)

Sky Medical Semi-Solid Ankle Foot Orthosis (SM-SAFO)

APPLICATION INSTRUCTIONS

Item Number	Size LEFT & RIGHT	Men's Shoe Size	Women's Shoe Size
SM-AFO-S	SMALL	5 – 6	6.5 – 7.5
SM-AFO-M	MEDIUM	6.5 - 10	8 – 11.5
SM-AFO-L	LARGE	10.5 - 13	12 – 14.5
SM-AFO-XL	X-LARGE	13+	14.5+
SM-SAFO-S	SMALL	5 – 6	6.5 – 7.5
SM-SAFO-M	MEDIUM	6.5 - 10	8 – 11.5
SM-SAFO-L	LARGE	10.5 - 13	12 – 14.5

Notice: This device is intended for use only under the direction of a medical professional. If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.

How to use:

1. Sit down in a chair. Place the foot into the brace.
2. Wrap the calf strap around the leg.
3. Insert foot in a lace up shoe.

Use and Care:

- Remove the brace while seated.
- The brace may be washed using a mild detergent in warm water. Rinse thoroughly and towel dry.



Maramed Orthopedic Systems
2480 West 82nd Street, #1A
Hialeah, Florida 33016 USA
T: (305) 823-8300
F: (305) 823-8304
E: custsupport@maramed.com