

# Deluxe Drop-Arm Commode

EN

REF	Description	Max User Weight
081532951	Deluxe Drop-Arm Commode	300 lbs 136 kg

The following manual provides descriptions relating to the installation of the Deluxe Drop-Arm Commode.

## Intended Use:

The Deluxe Drop-Arm Commode offers versatility for all users. The drop arm design allows for a smooth side transfer. Weight capacity limit of 300lbs/136kg and is height adjustable from 20.5 to 24.5" (seat height).

## Installation Instructions:

NOTE: Before assembly, ensure that all components are present. Inspect all components for damage. If any components are determined to be missing or damaged, contact your dealer/retailer for replacements.

WARNING! DO NOT use this commode without first reading and understanding the user manual. Ensuring the commode is suitable for the intended user and the environment of use.

## Assembly Instructions:

1. Please remove all packaging, taking care not to damage any of the components.
2. Taking one side of the arm frame, insert the support tubes ensuring that the pins have locked into place (you will hear a clicking noise) once the pins have fully protruded through the holes.
3. Repeat step 2 with the other side of the arm frame.
4. Insert the back into the slots, which are at the rear of the seat, and insert back until fully engaged and the pins are fully protruded.
5. The commode comes with a pail and lid for use as a commode or a splash guard for use over the toilet. Select and place on the underneath rim to secure in position.
6. Place the padded seat onto the support tubes, ensuring all four plastic clamps are secured on the frame.
7. To secure the armrests in position, rotate until the armrests are at the side of the commode and the armrest pin is locked into place (you will hear a clicking noise) once the pins have fully protruded through the holes.
8. Ensure when fully assembled, the commodes construction reflects Figure 1.

## CAUTIONS

- Ensure all four legs are adjusted to the correct height chosen for the intended user.
- Ensure all pins have locked into place (you will hear a clicking noise) once the pins have fully protruded through the holes.
- Ensure the seat is secure before applying any weight.
- The arm supports are only to be used for assistance. Not intended to support the full body weight of the user.
- Ensure both armrests are securely in position before using for assistance.
- Only use the commode for its intended use. They are not replacements for ladders and are not exercise equipment or toys.
- Under no circumstances should you stand on the commode.
- The commode is not intended for use during examinations or treatments.
- Use only spare parts and accessories released by Performance Health.
- Do not carry out structural changes which are not approved by the manufacturer.



## Cleaning:

- Small stains and dust are best removed with damp cloth. Please ensure the commode is dry after cleaning.
- Do not use tools with sharp edges (knives, metal scrapers) or aggressive solvents for cleaning.

## Maintenance:

The deluxe commode is maintenance-free, however, it should be checked on a daily basis for damage or looseness that could/may occur.

## Carry out the following test

- Visual check of all parts for deformation, damage, wear and tear and corrosion. The commode must not be used further if faults/defects are detected during use.

## Disposal:

The deluxe drop-arm commode consists of metal and plastic parts. Disposal of these materials must be carried out according to the regional environmental and disposal provisions.

## Restrictions of use:

- Patients weighing more than 300lbs/136kg must not use the Deluxe Drop-Arm Commode

## Limited Product Warranty

Performance Health Supply, LLC warrants that all products shall be free from manufacturing defects for a period of one year from the date of shipment. OTHER THAN THE FOREGOING WARRANTY, PERFORMANCE HEALTH SUPPLY, LLC MAKES NO REPRESENTATION OR WARRANTY OF ANY KIND, EXPRESS OR IMPLIED, AS TO MERCHANTABILITY, FITNESS FOR A PARTICULAR PURPOSE OR ANY OTHER MATTER RELATING TO THE PRODUCT. Customer must notify Performance Health Supply, LLC of any manufacturing defect within five days of discovery of the defect. Upon a determination by Performance Health Supply, LLC that the product is defective, Performance Health Supply, LLC sole and exclusive obligation shall be to provide customer with a replacement product or a full refund of the purchase price, at the sole discretion of Performance Health Supply, LLC, and to reimburse customer for any return freight costs. In no event shall Performance Health Supply, LLC be liable for any other costs or expenses including lost profits or direct, indirect, consequential, incidental or punitive damages. The warranty set forth herein shall govern all purchases, notwithstanding any inconsistent terms and conditions of customer's purchase order.

**Serious events experienced from using this product should be reported immediately to Performance Health and to the local Competent Authority.**

Distributed in the USA By:  
Performance Health Supply LLC  
W68 N158 Evergreen Blvd  
Cedarburg, WI 53012, USA  
+1800-323-5547  
www.performancehealth.com

**PH**  
**PERFORMANCE**  
**HEALTH®**