

30 Degree Bed Wedge

EN

REF	Description	Units
081334218	30 Degree Bed Wedge, 7" x 12" x 24", Smooth Surface	Pair
081508886	30 Degree Bed Wedge, 7" x 12" x 34", Smooth Surface	Each
081677129	30 Degree Bed Wedge, 7" x 12" x 24", High Density	Pair
081677137	30 Degree Bed Wedge, 7" x 12" x 34", High Density	Pair
081508928	30 Degree Bed Wedge, 7" x 12" x 24", Convoluted Surface	Each
081508910	30 Degree Bed Wedge, 7" x 12" x 34", Convoluted Surface	Each

Instructions for Use:

1. Positioning the User:

- Place the user on their side and slide the tapered edge of the wedge beneath them, positioning it just below the shoulders.
- Ensure that the non-slip matting faces the bed and does not directly contact the user's skin.

2. Transitioning onto the Wedge:

- Gently guide the user to roll back onto the wedge, allowing them to settle comfortably.

Cleaning Instructions:

Cushion can be surface cleaned using a damp cloth and regular liquid detergent. Use non-chlorine disinfectant only.

Do not immerse in water.

Do not steam clean or autoclave the product. Do not use abrasive cleaners. Allow to air dry completely.

Serious events experienced from using this product should be reported immediately to Performance Health and to the local Competent Authority.

Cale-lit à 30 degrés

FR

REF	Description	Unités
081334218	Cale-lit à 30 degrés, 7 po x 12 po x 24 po (17,8 cm x 30,5 cm x 61 cm), surface lisse	Paire
081508886	Cale-lit à 30 degrés, 7 po x 12 po x 34 po (17,8 cm x 30,5 cm x 86 cm), surface lisse	Chaque
081677129	Cale-lit à 30 degrés, 7 po x 12 po x 24 po (17,8 cm x 30,5 cm x 61 cm), haute densité	Paire
081677137	Cale-lit à 30 degrés, 7 po x 12 po x 34 po (17,8 cm x 30,5 cm x 86 cm), haute densité	Paire
081508928	Cale-lit à 30 degrés, 7 po x 12 po x 24 po (17,8 cm x 30,5 cm x 61 cm), surface convexe	Chaque
081508910	Cale-lit à 30 degrés, 7 po x 12 po x 34 po (17,8 cm x 30,5 cm x 86 cm), surface convexe	Chaque

Mode d'emploi :

1. Positionnement de l'utilisateur :

- Placez l'utilisateur sur le côté et faites glisser le bord conique de la cale sous eux, en la positionnant juste sous les épaules.
- Assurez-vous que le tapis antidérapant fait face au lit et n'entre pas directement en contact avec la peau de l'utilisateur.

2. Transition vers la cale :

- Guidez doucement l'utilisateur pour qu'il recule sur la cale, lui permettant de s'installer confortablement.

Instructions de nettoyage :

Le coussin peut être nettoyé en surface à l'aide d'un chiffon humide et d'un détergent liquide ordinaire. Utilisez uniquement un désinfectant sans chlore.

N'immergez pas dans l'eau.

Ne nettoyez pas le produit à la vapeur, ne le passez pas à l'autoclave. N'utilisez pas de nettoyants abrasifs. Laissez sécher complètement à l'air.

Tout effet indésirable grave dû à l'utilisation de ce produit doit être immédiatement signalé à Performance Health et à l'autorité locale compétente.

Distributed in the USA By:
Performance Health Supply LLC
W68 N158 Evergreen Blvd
Cedarburg, WI 53012, USA
+1800-323-5547
www.performancehealth.com

Distributed in Canada By:
Performance Health Canada, Inc.
2810 Argentia Road, Unit 2
Mississauga, ON L5N 8L2, Canada
+1800-665-9200
www.performancehealth.ca

