


Instructional guide:

How to put the
WristWidget® on left
or right wrist

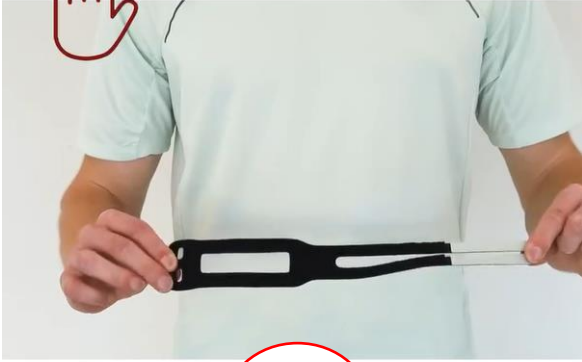


Step
1

Step 1: Remove WristWidget® from box and identify Velcro strips with tape.



Step
2



Step 2: Turn Velcro strips with tape facing downwards.

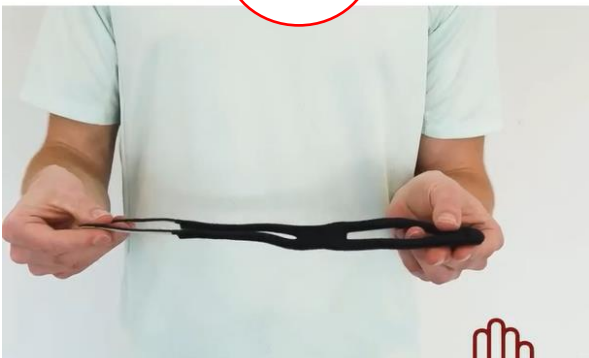




Step
3


Step 3: Insert the straps for the WristWidget[®] through the two openings.

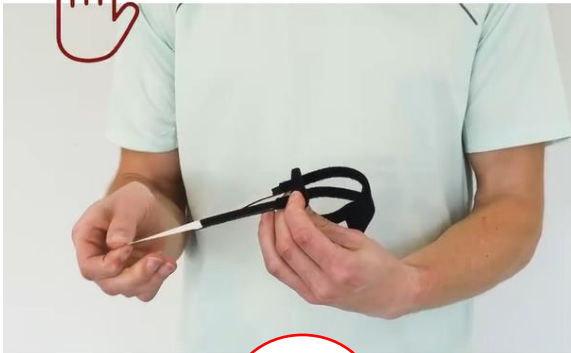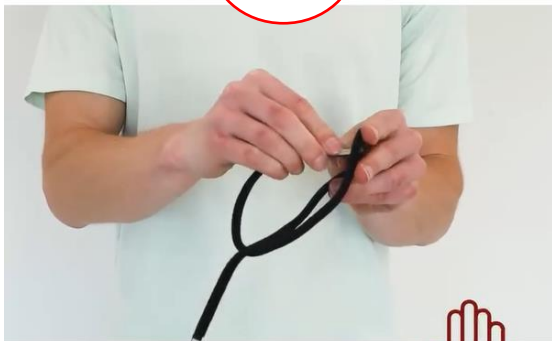

Step
4



Step 4: Remove tape.

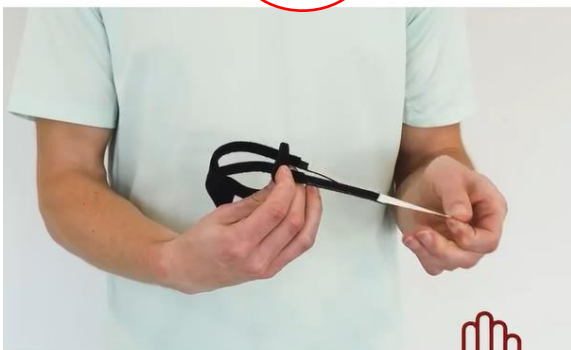

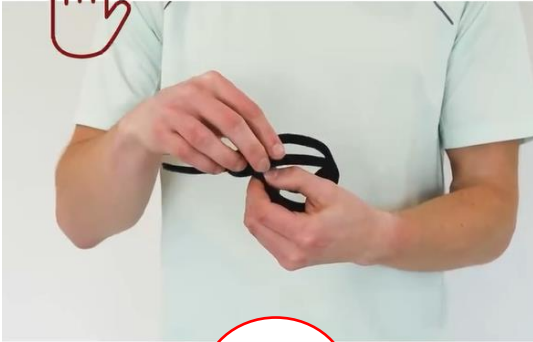


Step
5


Step 5: Stick Velcro straps down to hold in place.

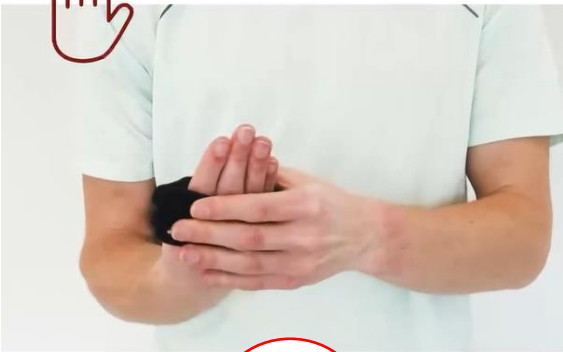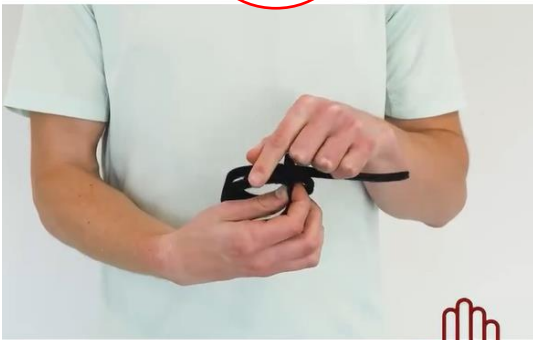

Step
6



Step 6: Place hand and wrist through the opening

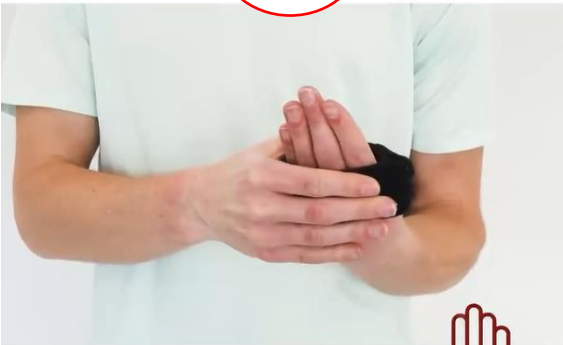

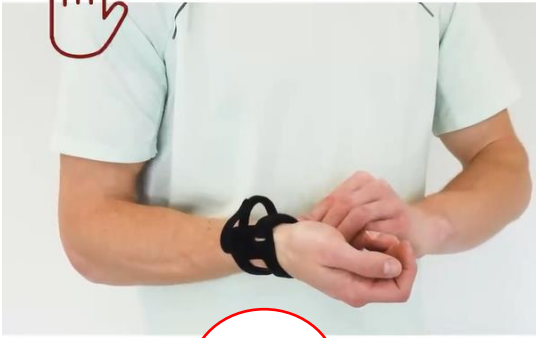

Step
7

Step 7: Adjust straps to tighten the WristWidget[®] to comfort.

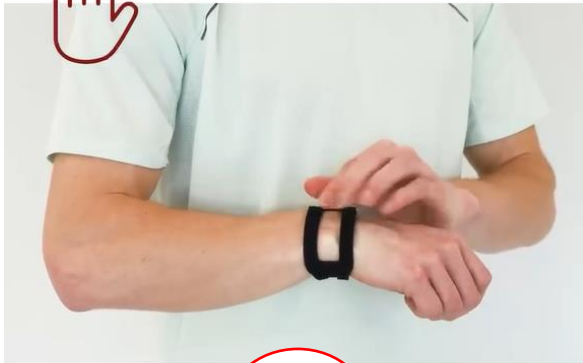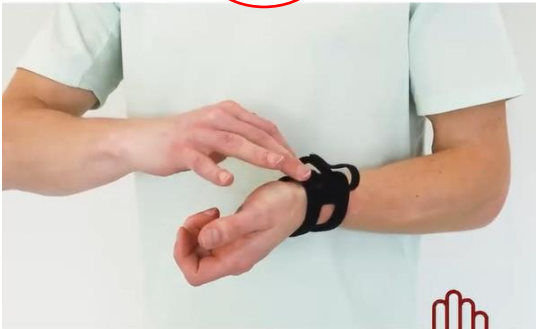

Step
8

Step 8: Make sure that the bony part is in the gap between the two straps.

