

HydraTherm Deluxe

CLEANING INSTRUCTIONS GUIDE



Thank you for purchasing the HydraTherm Deluxe. This Cleaning Instructions Guide will help you to get started quickly, but does not replace the Operating Manual. Be sure to read, understand, and practice the operating instructions, warnings, and cautions included in the manual.

Keep this instruction sheet as a quick-reference guide for cleaning your HydraTherm Deluxe. It is recommended to clean your unit a minimum of two times per year. The low water sensor may be wiped off more frequently to ensure cleanliness and proper function. Let water cool before wiping off sensor.

APPROVED CLEANING AGENTS:

When cleaning your HydraTherm Deluxe, please use a non-abrasive, low-to-no chlorine cleaner, such as a bleach-free multi-purpose cleaner. Certain additives such as herbal teas, essential oils, etc. that are added to cleaning solutions may damage the components of this heating device.

REMOVING HYDRAHEAT PACKS:

During extended periods of non-use, or when cleaning your device, lay HydraHeat Packs flat to maintain even distribution of gel inside the packs. This will make it easier to reintroduce packs

back into the tank. Do not store in areas with a temperature below 35°F/2°C.

CLEANING PREPARATION:

Turn the unit to the OFF position. Unplug the device. Let the water cool down to room temperature. Move HydraTherm Deluxe to a location where it may be drained.

- To use the electric drain pump, you must plug it into a properly grounded GFCI outlet to drain the water into a wastewater receptacle such as a sink, toilet, or floor drain

GATHER THE ITEMS NEEDED TO CLEAN THE HYDRATHERM DELUXE

- Cleaning solution made of any non-abrasive, bleach free, low to zero chlorine cleaner.
- Cleaning sponge with non-abrasive surface to clean packs, device, and remove debris where needed.

CLEANING STEPS ON BACK

HydraTherm Deluxe

CLEANING STEPS

STEP 1

Turn device off to let the water and packs cool to room temperature.



Device switch OFF

STEP 2

Remove packs from device and use sponge to clean with cleaning solution.

STEP 3

Remove Divider System to access bottom well of the HydraTherm Deluxe tank.

STEP 4

Remove any debris (e.g. fabric lint) from the bottom of the tank, including the well around the heating element, before beginning to drain the device.

⚠ Caution

DO NOT attempt to clean the debris from a tank with hot water. Please allow to cool to room temperature.

Manufactured for



Compass Health Brands Corp.
Toll Free 1.888.549.4945
6753 Engle Road
Middleburg Heights, OH 44130 USA
richmarweb.com

Made in Mexico

STEP 5

1. Position hose over wastewater receptacle.
2. Open valve by turning handle parallel with hose.
3. Change the device mode from HEAT to DRAIN.
4. Toggle the power switch to ON.
Water will begin emptying from the hose.
5. Once tank is empty, check the filter for debris by unscrewing to remove, clean out the debris and replace before closing the valve.



Valve OPEN



Device mode: DRAIN



Power switch: ON

STEP 6

1. When tank is empty, close valve by turning handle perpendicular to the drain hose.
2. Soak sponge in cleaning solution and wipe all surfaces of the HydraTherm Deluxe tank and dividers.
3. Rinse sponge and soak in water, and then rinse tank.
4. Remove any water and cleaning solution using appropriate draining method above.



Valve CLOSED

STEP 7

Refer to Quick Start Guide or Operation Manual for instructions on how to properly fill your device.