

How To Wear

These easy to apply therapeutic gloves have been specifically designed to help swelling of the hands. These gloves offer moderate compression, effectively reducing swelling and may alleviate pain. With seams on the external side, reducing any potential catching on the skin. Ensure during use, the seams are always on the external side of the gloves. Can be wore during the day and night.

Cleaning & Drying

Hand wash in cold water (86F/30C). Use a mild detergent suitable for delicate or elastic fabrics. Rinse thoroughly to remove all residue. Gently press to remove excess water. Reshape and lay flat to air dry naturally

Storage

Keep away from direct heat sources and/or sunlight when drying to prevent damage to the fabric. Store the gloves in a cool, dry place away from direct sunlight when not in use to maintain quality.

Caution

Ensure the therapeutic glove is suitable for the intended user, paying special attention to selecting the correct size

Should you experience any discomfort, irritation or develop a rash, immediately discontinue use and seek medical advice.

LIMITED PRODUCT WARRANTY

Performance Health Supply, LLC warrants that all products shall be free from manufacturing defects for a period of one year from the date of shipment. OTHER THAN THE FOREGOING WARRANTY, PERFORMANCE HEALTH SUPPLY, LLC MAKES NO REPRESENTATION OR WARRANTY OF ANY KIND, EXPRESS OR IMPLIED, AS TO MERCHANTABILITY, FITNESS FOR A PARTICULAR PURPOSE OR ANY OTHER MATTER RELATING TO THE PRODUCT. Customer must notify Performance Health Supply, LLC of any manufacturing defect within five days of discovery of the defect. Upon a determination by Performance Health Supply, LLC that the product is defective, Performance Health Supply, LLC sole and exclusive obligation shall be to provide customer with a replacement product or a full refund of the purchase price, at the sole discretion of Performance Health Supply, LLC, and to reimburse customer for any return freight costs. In no event shall Performance Health Supply, LLC be liable for any other costs or expenses including lost profits or direct, indirect, consequential, incidental or punitive damages. The warranty set forth herein shall govern all purchases, notwithstanding any inconsistent terms and conditions of customer's purchase order.



Performance Health Supply LLC

W68 N158 Evergreen Blvd

Cedarburg, WI 53012, USA

+1800-323-5547

www.performancehealth.com

Performance Health® is a registered trademark. ©2024 All rights reserved. Unauthorized use is strictly prohibited.

75657-R00