

**VICAIR<sup>®</sup>**

O2 cushions



**permobil**



## Vicair technology

Vicair Technology uses air filled "SmartCells™" – which offer a unique combination of skin protection and positioning. The SmartCells in each cushion compartment shift and deform individually and together to adjust to the body.

Vicair O2 cushion compartments are designed to provide stability and therefore positioning possibilities. Each compartment is filled with just the right amount of SmartCells and for most individuals the product is ready to use out of the box. However, in case of specific needs every compartment can also be adjusted to fit individual needs by removing and adding SmartCells™.

## Machine washable & breathable

Vicair Adjuster O2 and Vicair Vector O2 cushions can be put in the washing machine in their entirety. The SmartCells do not have to be removed during washing so your cushion set-up is maintained.

Vicair O2 cushions have an open structure that offers 100% breathability and ventilation. This helps with temperature and humidity control, increasing comfort and reducing the effects of shear and friction on tissue deformation.





# Vicair Adjuster O2

HCPCS E2622 & E2623  
Adjustable Skin Protection

## Adjuster O2 Standard

The Vicair Adjuster O2 Standard is an adjustable skin protection cushion that provides support for the IT/coccyx area, redistributes pressures over a larger surface area and may be adjusted to assist in managing pelvic asymmetry.

The front compartments offer femoral support and the pre-ischial shelf minimizes sliding forward. Adjuster O2 comes with a Comfair Cover or Incotec Cover and has a Kwik Strap® option to help keep the cushion secure to the wheelchair. This cushion also offers stability and comfort and the entire cushion is machine washable.



## Adjuster O2 Low

Vicair Adjuster O2 is available in two thicknesses: standard (4 inch) and low (2.5 inch). Adjuster O2 Low comes with an Incotec cover with Kwik Strap attachment to help keep the cushion secure to the wheelchair. This cushion also offers stability and comfort and the entire cushion is machine washable.

## Features

- Individual solution for asymmetry
- Minimizes sliding and shear forces
- Adjustment can be individualized to the body's contours for pressure redistribution
- Temperature and moisture regulation
- Fully machine washable

# Vicair Vector O2

HCPCS E2624 & E2625  
Adjustable Skin Protection & Positioning

## Adjustable positioning

The Vicair Vector O2 Standard is an adjustable skin protection and positioning cushion that provides greater trochanter support for lateral stability and redistributes pressure to help create an ischial well. The front compartments provide leg alignment and stability and can be adjusted for individual positioning needs.

The Vicair Vector O2 comes with a Comfair cover or Incotec cover and has a Kwik Strap option to help keep the cushion secure to the wheelchair. This cushion also offers stability and comfort and the entire cushion is machine washable.



## Features

- Nine adjustable compartments for stability
- Extra positioning of the thighs
- Expanding the individual's reach
- Temperature and moisture regulation
- Fully machine washable

# Vicair Active O2

HCPCS E2624 & E2625  
Adjustable Skin Protection & Positioning

## Hybrid air - foam

The Vicair Active O2 Low offers the unique combination of ergonomically shaped foam and Vicair SmartCells. The four rear compartments provide a high level of comfort, skin protection and reduce pressure in the IT/coccyx area.

The foam front assists in making transfers easier. The foam inlay is protected by an easy-to-clean fluid-resistant liner. The rest of the cushion is 100% machine washable and breathable for optimal hygiene, temperature and moisture regulation.



## Features

- Accessible storage pouch
- Great for lateral and frontal stability
- Ideal for individuals who transfer frequently
- Temperature and moisture regulation
- Machine washable (must remove foam)



## Specifications

  = Nonstock items available at extended lead times and a \$40.00 upcharge

### Vicair Adjuster O2 Standard & Vicair Vector O2 Standard

<b>SmartCell compartments</b>	5 — Adjuster O2   9 — Vector O2
<b>Cover options</b>	Comfair or Incotec
<b>Optional attachment*</b>	Kwik Strap on limited sizes
<b>Cushion thickness</b>	4"

				18"W 22"D	19"W 22"D	20"W 22"D	22"W 22"D	24"W 22"D
					19"W 21"D			
		16"W 20"D *	17"W 20"D	18"W 20"D *	19"W 20"D	20"W 20"D *	22"W 20"D *	24"W 20"D *
		16"W 19"D	17"W 19"D	18"W 19"D	19"W 19"D	20"W 19"D		
14"W 18"D	15"W 18"D	16"W 18"D *	17"W 18"D	18"W 18"D *	19"W 18"D	20"W 18"D *	22"W 18" D *	24"W 18"D *
		16"W 17"D	17"W 17"D	18"W 17"D		20"W 17"D		
14"W 16"D	15"W 16"D	16"W 16"D *	17"W 16"D	18"W 16"D *		20"W 16"D *		
	15"W 15"D	16"W 15"D						
14"W 14"D								

### Vicair Adjuster O2 Low

<b>SmartCell compartments</b>	5
<b>Cover</b>	Incotec with Kwik Strap
<b>Cushion thickness</b>	2.5"

16"W 20"D	18"W 20"D	20"W 20"D
16"W 18"D	18"W 18"D	20"W 18"D
16"W 16"D	18"W 16"D	20"W 16"D

### Vicair Active O2 Low

<b>SmartCell compartments</b>	4
<b>Cover</b>	Integrated mesh fabric
<b>Cushion thickness</b>	2.5"

		18"W 22"D	20"W 22"D	22"W 22"D	24"W 22"D
	16"W 20"D	18"W 20"D	20"W 20"D	22"W 20"D	24"W 20"D
14"W 18"D	16"W 18"D	18"W 18"D	20"W 18"D	22"W 18" D	
14"W 16"D	16"W 16"D	18"W 16"D	20"W 16"D		
14"W 14"D					

*Comfair cover - This breathable cover is made out of multi-dimensional stretch material, reducing pressure on bony prominences. The breathable fabric is designed for air transmission and heat dissipation.*

*Incotec cover - A high-quality fluid resistant cover made out of elastic material. Extremely usable in all situations (institutional and home use) where incontinence and hygiene is an issue.*

*Permobil is the U.S. distributor of Vicair O2 cushions.*

For ordering or customer support contact **800.736.0925** or [orders.comfort@permobil.com](mailto:orders.comfort@permobil.com)