

IMAK SMART GLOVE

PATTERSON PART NUMBER:

081254390

081254416

081254432

Instructions for Use:

1. Pull on SmartGlove with moldable pad centered under palm and the splint insert on top of hand.
2. Adjust pad by moving ergoBead filling to desired position.
3. SmartGlove can be worn with or without splint. The splint may be removed at the channel opening. For best results, you may want to try it both ways to determine your own level of comfort and need.
4. The splint is made of flexible plastic to allow your hand to function fully. If worn with the splint, be sure it is pushed all the way to the front of the splint channel (under the SmartGlove label)

To Wear on Opposite Hand: turn SmartGlove inside out.

Care Instructions:

Remove splint, hand wash, air dry.