

## IMAK SMART GLOVE WITH THUMB

PATTERSON PART NUMBER:

081254390

081254416

081254432

### Instructions for Use:

1. Pull on SmartGlove with moldable pad centered under palm, the dorsal splint insert on top of hand, and the thumb into the thumb support.
2. Adjust pad by moving ergoBead filling to desired position.
3. SmartGlove can be worn with or without the dorsal splint. The splint may be removed at the channel opening. For best results, you may want to try it both ways to determine your own level of comfort and need.
4. The splint is made of flexible plastic to allow your hand to function fully. If worn with the splint, be sure it is pushed all the way to the front of the splint channel (under the SmartGlove label)

*To Wear on Opposite Hand: turn SmartGlove and Thumb Support inside out.*

### Care Instructions:

Remove splint, machine wash in cold water; tumble dry cool.