Indications

- Chronic weak ankles, post ankle trauma, post-op ankle recovery, ankle support, prophylactic ankle protection

Contraindications

- Fractures, open wounds

Cleaning Instructions/Maintenance**REMOVE RIGID STAYS BEFORE WASHING**

- The Web™ Ankle Brace is machine washable using a mild detergent and warm water.
- To prevent the hook and loop straps from damaging other items in the wash, place the Web™ Ankle Brace in the included mesh bag before washing or wash separately.
- Remove promptly and allow to air dry only.

Instructions for use

1. Lift flaps from stay pockets located on each side of the ankle brace. Insert the Lateral stay (Red sticker) into the outside pocket. Insert the Medial stay (Blue sticker) into the inside pocket. The stays may be switched to accommodate a left or right ankle.
2. An adhesive backed foam spacer is included with the brace. The spacer can be trimmed and placed on the rigid stays if more padding is needed for comfort.
3. Re-insert the flaps over the stays to secure them in place. Note: If less rigidity is required this step can be skipped or the stays can be removed at a later time.
4. Open up the back of the brace, slide your foot into the brace and secure the hook and loop fastener backing evenly.
5. Stretch the lower straps equally and wrap them around your ankle, securing them in the back.
6. Cross the upper strap over the center and secure around the top of ankle.

Caution

Please always follow these instructions for use. In case of pain or swelling, discontinue use and consult your medical provider immediately. This device is designed to assist recovery and should be used exclusively under the instruction of and in consultation with a healthcare professional. DARCO International Inc. does not assume any responsibility for misuse of the product. It is advisable to wear socks with the Web™ Ankle Brace. Do not drive a vehicle while you are wearing this device. Use extreme caution when walking on all surfaces when using this device. Walk more slowly than normal. This device is for single-patient use only.

To the patient/user: This device has been designed and manufactured by DARCO International, Inc. to be state of the art for treatment of a specific medical condition. Should any serious incident occur in relation to this device it should be reported to the manufacturer at the contact information listed on this document and the proper authority of the country or state where you are located. When treatment is complete, dispose of this device according to local laws and ordinances.

UK
CA**Manufactured for:**

DARCO International, Inc.
810 Memorial Blvd.
Huntington, WV 25701, USA
Phone 304.522.4883

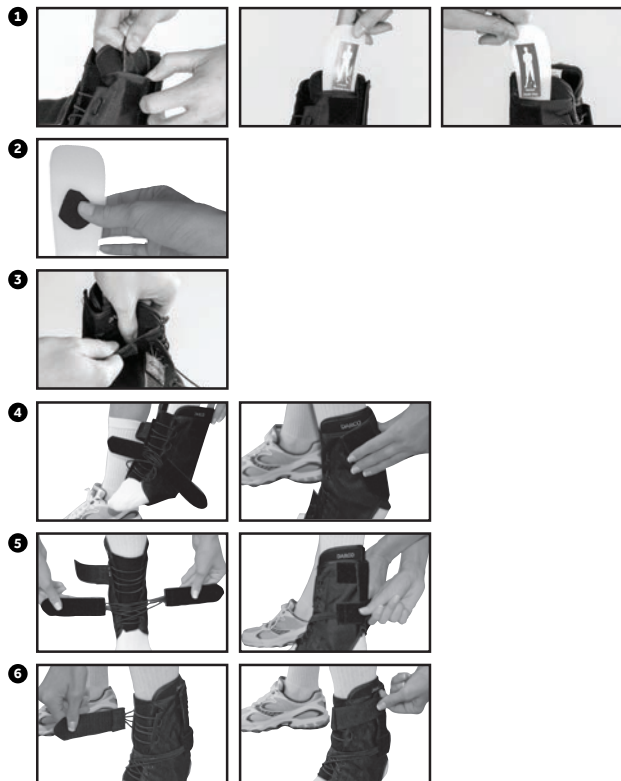
UK Responsible Person:

V-M Orthotics Ltd.
Unit 25 Halesworth Business
Centre Norwich Road
Halesworth Suffolk IP19 8QJ
United Kingdom

EC REP

DARCO (Europe) GmbH
Gewerbegebiet 18
82399 Raisting, Germany
Telefon +49 8807 9228 0
www.darco.de

DARCO**Web™ Ankle Brace****Instructions For Use**

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