



Available with:
Smooth or Convuluted Foam interior



HEELIFT Boots

APPLICATION OF HEELIFT[®]

1

Measure for correct sizing. Place the foot inside of boot with the heel above the heel opening.



2

Thread each strap through its respective D-ring and secure.



3

Using the two-finger test, check each strap to make sure the straps are fastened appropriately so that the boot is not too tight or not too loose. Check to confirm the heel is completely off-loaded by looking in the heel opening.



PROPER SIZING



CALF CIRCUMFERENCE:

HEELIFT [®] CLASSIC	SIZE	Smooth:	Convuluted:	Each
STANDARD	10-15 in (25-38 cm)	081194398	081194380	Each
		081194398	081194380	Each
PETITE	6-10 in (15-25 cm)	081422690	081433044	Each
		081422690	081433044	Each
BARIATRIC	15-23 in (38-58 cm)	081422708		Each

WHEN APPLYING THE HEELIFT[®] BOOT:

- Determine the appropriate size (see measuring guide above).
- Check to make sure the heel is completely offloaded by:
 - Lifting the leg and looking in the heel opening to ensure the heel is completely offloaded and no pressure is on the back of the heel.
 - Put hand through the heel opening and cup the heel to ensure the heel is completely offloaded and no pressure is on the back of the heel.
- Check that straps are not too tight or too loose by performing the two-finger test.
 - *NOTE: Make sure the straps are not touching the skin.
- Review the algorithm or facility criteria for application.
- All HEELIFT[®] boots may be used on the right or left foot.

*NOTE: As per standard of care, remove HEELIFT[®] boot and inspect skin every 8 hours to ensure no compromise of the skin has occurred.

*SINGLE PATIENT - MULTIPLE USE, NOT MADE WITH NATURAL RUBBER LATEX

OPTIONAL CUSTOMIZATION

Application and customization of the boot should only be performed per the discretion of the healthcare provider after reviewing IFU cautions and warnings.

*Consult Instructions for Use for Customization and Cleaning Instructions



ACHILLES TENDON



MALLEOLI



FOOT DROP



SCD TUBING FOR DVT PREVENTION



FOOT ROTATION

For the prevention of internal and external foot rotation, apply the optional HEELIFT[®] Anti-Rotation Wedge (sold separately).
REF NO: 831727009898



Suggested HCPCS: E0191

Distributed by:



Phone: (800) 323-5547
Website: www.performancehealth.com