



THE COMFORT COMPANY

Quietly making lives more comfortable™

LATERAL SUPPORT ASSEMBLY



1 Remove harness and lateral pads from package.

2 Separate lateral pads from harness.

3 Attach harness to wheelchair backrest.

a) Wrap the single strap around the top/bottom of the wheelchair backrest. Secure using the hook/loop.

i. **Note: There is an additional strap included in the package for tall wheelchair backrests (i.e. reclining wheelchairs).**

b) Wrap the horizontal straps around the outside of the wheelchair. Secure using the hook/loop straps.

4 Re-attach the lateral pads to the harness in the desired location.