

Titan Wrist

Patient

Warnings and Instructions: Review carefully, proper application is required

⚠ Warning: This device will not prevent or eliminate risk of injury. Do Not Over-tighten. If swelling, pain, skin irritation, or an unusual reaction occurs, discontinue use immediately and consult your medical professional.

Care: Hand wash using mild soap. Rinse thoroughly. Air dry only. Do not tumble dry.



1. Position hand in brace.



3. Secure the thumb strap to the brace between the thumb and index fingers.



1a. Make sure the palm stay lines up below the midhand crease.



4. Wrap the lacing strap to Titan™ the wrist brace.



2. Align the lacing stay pod with the middle knuckle as shown, making sure that the wrist brace does not extend over the knuckles.



5. The Titan™ Wrist should fit below the mid hand crease.



5a. It should not obstruct the knuckles.

HELY & WEBER

© 2007 Weber Orthopedic Inc.
DBA Hely & Weber All rights reserved.
1185 East Main St., Santa Paula, California 93060
California: 800-221-5465, National: 800-654-3241
International: 805-525-4244, fax: 805-933-2348
U.S. fax: 800-559-5975, www.hely-weber.com

CE

EC REP

MDSS GmbH
Schiffgraben 41
30175 Hannover, Germany

Patent No: 6893410, 6960176