
# CLEANPatch-V®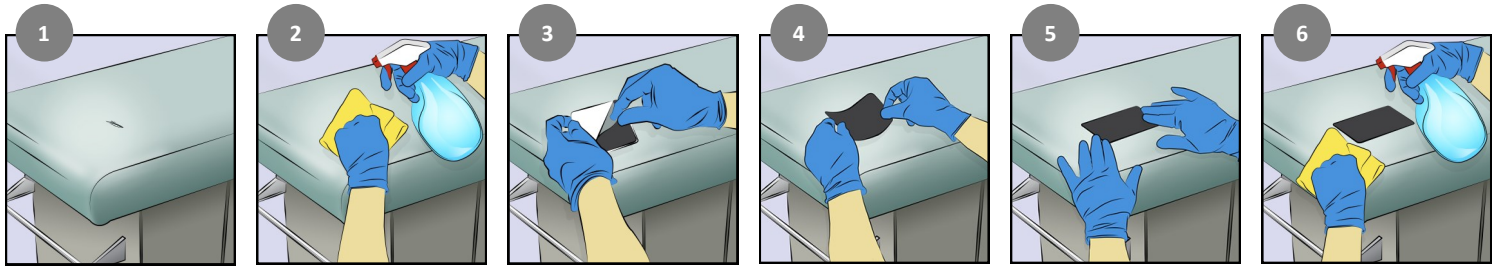

## INDICATION FOR USE

CleanPatch-V is a medical surface repair patch for vinyl upholstered equipment such as exam tables, wheelchairs, treatment tables, patient chairs and other vinyl surfaces. CleanPatch-V restores damaged vinyl pads to an intact and hygienic state. The product is not designed, sold or intended for use except as indicated.

## PRECAUTIONS

- Unclean or wet surfaces will significantly reduce CleanPatch-V performance
- For best results, apply CleanPatch-V at room temperature
- CleanPatch-V is intended for one-time application only. Do not try to re-apply
- Immediately remove and replace CleanPatch-V if edges are lifting
- If tear, cut, or puncture extends beyond CleanPatch-V, remove and replace with a larger size
- Layering multiple patches to cover a large area is not recommended
- CleanPatch-V is suitable for flat surfaces and gently rounded edges
- Not suitable for application over 3-dimensional corners
- To ensure product integrity do not cut CleanPatch-V

## DIRECTIONS FOR USE

### Before Applying CleanPatch-V

1. Clean the surface thoroughly using applicable cleaning guidelines (Figure 2)
2. Allow the surface to dry completely
3. Use the appropriate size of CleanPatch-V to allow sufficient intact margin around the damage

Ref #	Size	Damage Size
#21001 (10/box)	10 x 4.5cm/4 x 1.75in strip (S)	Damage up to 5 cm (2 in)
#22001 (5/box)	20 x 15cm/8 x 5.75in rectangle (R)	Damage up to 15 cm (6 in)
#23001 (10/box)	7 (S) + 3 (R)	Damage up to 15 cm (6 in)
#24001 (3/box)	2 (S) + 1 (R)	Damage up to 15 cm (6 in)

### Applying CleanPatch-V

1. Ensure that your hands or gloves are clean during application
2. Peel the white bottom liner exposing the adhesive surface (Figure 3)
3. Position and center CleanPatch-V over tear, cut, or puncture (Figure 4)
4. Apply CleanPatch-V allowing for good contact with undamaged surface
5. Apply firm pressure around the sides and corners to enhance adhesion (Figure 5)
6. Avoid applying CleanPatch-V over seams

### After Applying CleanPatch-V

1. Clean the patch and repaired area using applicable cleaning guidelines (Figure 6)
2. Date the patch with a non-toxic Sharpie® marker to help track repair

### Cleaning Instructions

- Follow applicable hospital guidelines for cleaning vinyl surfaces
- Use of a non-linting wipe is strongly recommended

## STORAGE AND SHELF LIFE

- For best results, store in a cool dry place.
- Avoid excessive heat, humidity, and direct sunlight
- For shelf life, refer to the "Use By" date printed on each package

If you have any questions or comments contact the Surface Medical Customer Helpline: **1-888-623-7085** or visit [www.cleanpatch.ca](http://www.cleanpatch.ca)

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