Material
Schaft:
Polyamid (PA)
Polyester (PES)
Polyurethan (PUR)
Futter und Decksohlenfutter:
Polyester (PES)
Polyurethan (PUR)
Sohle:
Polyurethan (PUR)

Material
Upper:
Polyamide (PA)
Polyester (PES)
Polyurethane (PUR)
Upper- and foot plate lining:
Polyester (PES)
Polyurethane (PUR)
Sole:
Polyurethane (PUR)

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BAUERFEIND.COM

Indications/Areas of Application

- For post-operative and/or post-traumatic conditions of the forefoot area
- For diabetic/neuropathic plantar ulcers in the forefoot area
- For rheumatic feet and algodystrophic syndromes

Contraindications/Application Risks

Mobility and balance may be compromised based on the purpose and desired function of the **forefoot-relieving orthosis**. To date, there are no known side effects. Correct usage and fitting by a trained specialist is assumed. To date, there are no known hypersensitivities that cause illnesses. This product should be fitted and worn only after consulting your physician in the following clinical situations:

- Skin diseases/injuries in the body part being treated, particularly if inflammation is present.
- Likewise, any raised scars with swelling, redness, and excessive heat build-up.
- Acroparesthesia in the body parts being treated
- Compromised lymph system, circulation problems, unclear soft tissue inflammation around the product's area of application

Cleaning Instructions

Be sure to have your **"Postop" forefoot-relieving orthosis** checked regularly for wear and tear and to ensure it is working properly by an orthotist if necessary, exchange the device. We recommend washing the orthosis separately at 40° C by hand using a delicate detergent. We recommend closing the Velcro fastenings before washing in order to preserve their functionality and avoid damage. The orthosis should not be exposed to direct heat when drying it (i.e. heating vents, sunlight, etc.). This product has been tested by our in-house quality management system. However, should you have any complaint, please contact our local expert adviser or our customer service department.

Safety Instructions

Any open wounds on the foot must be protected by a bandage or similar device. Do not ride a bicycle or drive a car or any other vehicle while wearing the orthosis. Use crutches when wearing a **forefoot-relieving orthosis**. Wearing the **"Postop" forefoot-relieving orthosis** is only recommended in hospitals and clinics or within your own home but not in public areas. Please see your physician immediately should you notice any irregular changes in your body (i.e. increase in symptoms). Do not alter the product in any way as it may affect the product's functionality in such a way that product liability cannot be assumed.

Application Instructions

This product is intended for the treatment of one individual patient and may not be reused or used by others. Product liability cannot be assumed in the event of inappropriate use. The **forefoot-relieving orthosis** should be fitted while the patient is sitting and can be worn both on the left and right foot. Secure the Velcro fastenings so it fits snugly against the foot, although not too tight in order to avoid pressure points on the skin or damage to arteries and nerves. Always wear a non-skid shoe with the same heel height on your healthy foot. If necessary, a physician may prescribe that the height of the heel be changed by an orthotist.

Warranty

The legal provisions of the country in which the product was purchased apply. Country-specific warranty regulations that govern the relationship between the retailer and the buyer also apply where relevant. If you believe that a claim may be made under the warranty, please first contact the retailer from which you purchased the product directly. Please do not make any changes to the product yourself. This particularly applies to individual adjustments made by specialists. Please follow our instructions for use and care. These are based on our experience gained over many years and ensure the functionality of our medically effective products for a long time. Only aids that function optimally can help you. Failure to follow the instructions may also restrict the warranty. First-time application and fitting of the forefoot-relieving orthosis may only be carried out by trained specialists, otherwise product liability cannot be assumed.

Disposal

You can dispose of the product in accordance with national legal requirements.

Version: 2015-05

en english

Dear Customer,

thank you for choosing a product from Bauerfeind. **GloboPed** is a high-quality medical product. Please read through these instructions for use carefully and if you have any questions contact your doctor, medical retailer or our Technical Service department.

Product Description

The **"Postop" forefoot-relieving orthosis** is a prescribable product that may only be worn under a physician's direction and only in accordance with these instructions for use and for the applications named. The **"Postop" forefoot-relieving orthosis** may only be worn with a correctly installed protective frame that protects the forefoot from mechanical influence.

Technical Data

Model	Size	Shoe Size	Article Number
Postop	S	33-34	3 662700 001
	M	35-38	3 662700 002
	L	39-42	3 662700 003
	XL	43-46	3 662700 004

Accessories

Model	Size	Article Number
Protective Frame, Black Plastic	S+M	3 662490 002
	L+XL	3 662490 003
Protective Frame, Aluminum	S-XL	3 662490 001

Purpose

The **"Postop" forefoot-relieving orthosis** is a high-quality orthopedic product for targeted relief of the area around the forefoot and front metatarsal area.