



- 1501 TERRY-CLOTH & PLASTIC HAND CONE**
- 1495 CONE AND FINGER SEPARATOR**
- 1494 FINGER CONTRACTURE CUSHION (Regular)**
- 1494-01 FINGER CONTRACTURE CUSHION (Large)**

MONITORING

▲WARNING

Be sure to follow your facility's policies and procedures for frequency of patient monitoring. We recommend that this product be removed at least every few hours to check for skin integrity, proper circulation and rehabilitation exercises.


INSPECTION

▲WARNING

Inspect before each use. Check for broken stitches; or torn, cut or frayed material; DO NOT use soiled or damaged products.

TERRY-CLOTH & PLASTIC HAND CONE

With 1" elastic strap for snug, comfortable fit. Removable, washable cover. Measures 5" x 1" x 1 1/2". Latex free. **(1501)**

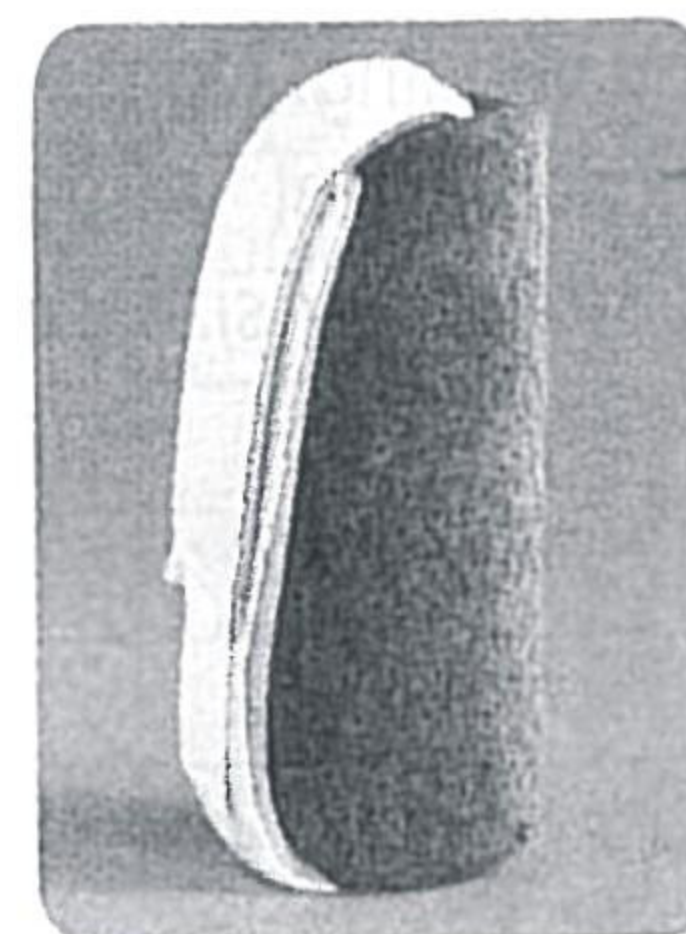

(1501)

CONE AND FINGER SEPARATOR

Firm coneshaped anti-contracture device conforms to hand's natural shape, keeps fingers separated to prevent skin breakdown. Plush cotton terry cover absorbs perspiration. Adjustable strap fits all hand sizes, holds cone securely. Machine washable. Latex free. **(1495)**




(1495)

FINGER CONTRACTURE CUSHION

Separates fingers to help prevent complete closure of the hand and protect palms. Polyester-filled cushion; cotton terry cloth absorbs perspiration. Secures to client's hand with 1/4" wide elastic strap. Washable. Packages of 3. Latex free.

(1494) Regular, 3" x 5" (7.6-12.7cm)

(1494-01) Large, 4" x 5" (10.2-12.7cm)

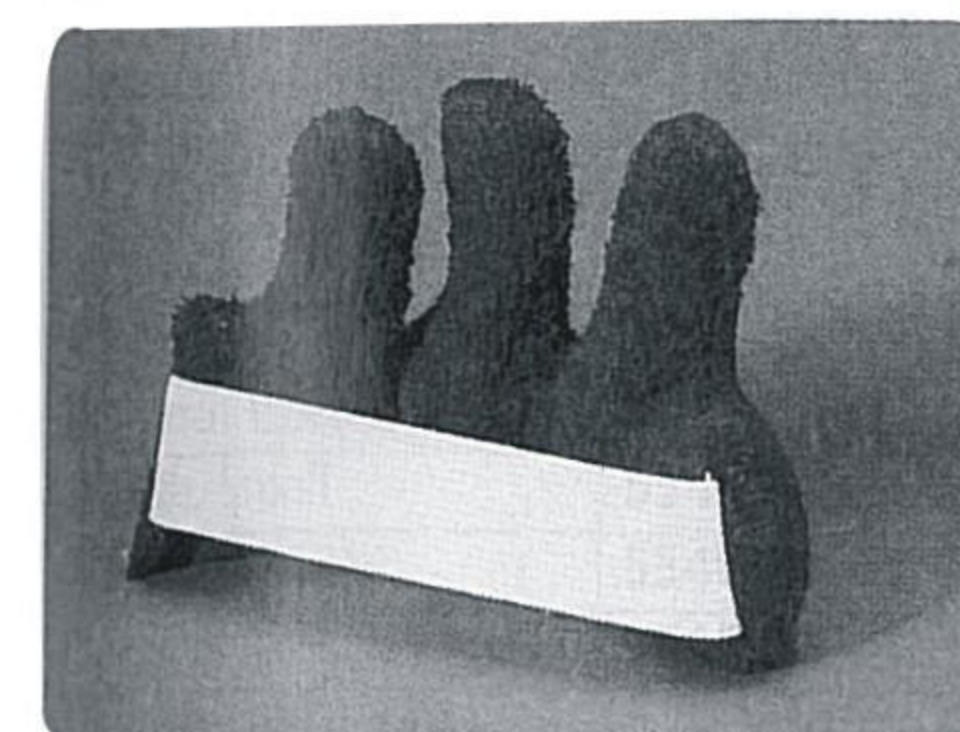

(1494)
(1494-01)

WASHING INSTRUCTIONS



DISPOSAL

▲WARNING Properly dispose of the product per facility's policy for BIOHAZARDOUS materials.

STORAGE AND HANDLING

This device is designed for use in normal indoor environments. This device may be stored in ambient warehouse temperatures at normal humidity levels. Avoid excess moisture or high humidity that may damage product materials.



Patterson Medical
1000 Remington Blvd.
Suite 210
Bolingbrook, IL 60440
Phone: 800-323-5547
Direct Phone: 630-378-6000
Fax: 800-547-4333
Direct Fax: 630-378-6310