

Rolyan®
a Patterson Brand



Graded ROM Arc

Item Sku# 5291

Item Description: Ideal for treatment of any upper extremity deficit that impairs range of motion (ROM). Arc easily adjusts to 15", 20", 25" and 30" heights. At 36" x 3/4" x 6", the natural wood base is portable for clinic or home use. Plastic tube is 3/4". Includes 12 rings. Latex free.

Indications:

Ideal treatment for any upper extremity deficit that impairs range of motion (ROM). The arc includes 12 movable rings for ROM exercises such as crossing the midline, eye and hand coordination and pinch strengthening.

Care and Cleaning:

Rolyan Graded ROM Arc can be wiped clean with a mild detergent – no bleach, no heavy duty cleaning products - to clean. Do not submerge in water. Dry with soft cloth.

Procedures:

To use, client moves brightly colored rings from side to side, developing fine motor, crossing-midline and eye-hand coordination. Wrist weights can be added to strengthen arms, hands and wrists. Weights sold separately.



Distributed by:

Patterson Medical
28100 Torch Parkway Ste. 700
Warrenville, IL 60555, USA
Phone: 800-323-5547
Direct Phone: 630-393-6000
Fax: 800-547-4333
Direct Fax: 630-393-7600

Patterson Medical
W68 N158 Evergreen Blvd
Cedarburg, WI 53012, USA
Phone: 800-323-5547
Fax: 800-547-4333

Patterson Medical Canada
6675 Mill Creek Drive, Unit 3
Mississauga, ON L5N 5M4
Phone: 800-665-9200
Direct Phone: 905-858-6000
Direct Fax: 905-858-6001

**Authorized European Representative
Patterson Medical, LTD.**
Nunn Brook Road
Huthwaite, Sutton-in-Ashfield
Nottinghamshire, NG17 2HU, UK
International
Phone: +44 (0) 1623 448 706
Fax: +44 (0) 1623 448 784