
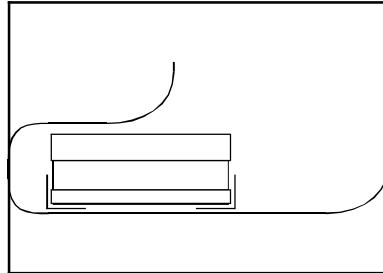


# ECONOMY ARM CHANNEL BRACKET

## MOUNTING BRACKETS TO TRAY

---

1. Slide both brackets onto wheelchair armrests. Rest tray atop brackets in desired position. Mark where edges of ONE bracket makes contact with tray.
2. Remove tray and place upside down on a flat working surface. Make sure area where brackets will be placed is clean and dry.
3. Remove protective layer of tape, exposing the sticky tape. Position bracket on tray, lining up with marks. Apply pressure to entire bracket area to make sure it is solidly attached to tray.
4. Reposition tray onto wheelchair and repeat steps with the other bracket.

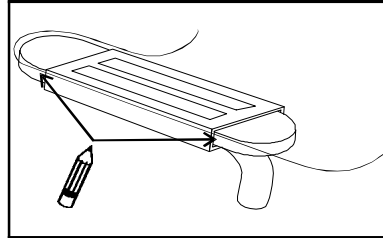

08822



9450 W Laraway Rd • Frankfort, IL 60423  
Customer 800-843-7234 • Fax 888-479-1515  
[www.TheraFin.com](http://www.TheraFin.com) • [info@TheraFin.com](mailto:info@TheraFin.com)